
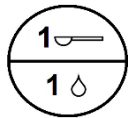


dline Glass Ionomer Base Lining Cement

Glass ionomer base lining cement for base linings under composite restorations and amalgams

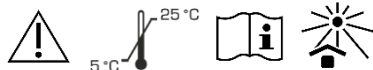

EN ISO 9917-1 4.2b Glass Polyalkenoate Cement

INSTRUCTION FOR USE EN

DESCRIPTION

dline Glass Ionomer Base Lining Cement is a chemical curing, fast-setting, radiopaque glass ionomer cement with excellent thermal expansion and high strength and hardness.

INSTRUCTION FOR USE

NOTE! Do not use if primary package is broken.

1. Prepare surface as always. For deep cavities use calcium hydroxide liner.
2. Mixing ratio: 1 level spoon of powder / 1 drop of liquid. Lightly tap the bottle of powder against the hand before dispensing. Do not shake or invert. Hold liquid bottle vertically upright and squeeze gently when dispense a liquid.
3. For mixing use metal or plastic spatula and glass slab. Divide powder into two parts. Mix the first half of powder with liquid for 10 seconds, and then add remainder. Mix until exact consistency is reached. Total mixing time is 20 seconds.
NOTE! As dline Glass Ionomer Base Lining Cement highly adhesive, clean all instruments before cement set on.
4. The working area must be kept dry. Apply the mixed cement with suitable instrument into cavity. Total working time is 2 minute including mixing.
5. After 6 minutes from start of mixing lining is ready for continue restoration.
6. Higher amount of powder or higher temperature shorten the working/setting time and lower temperature, or higher amount of liquid prolong the working/setting time.

WARNINGS AND PRECAUTIONS!

NOTE! Do not use dline Glass Ionomer Base Lining Cement if patient is allergic to product or to any of the ingredients.

Hazard and precautionary statements:

Wear protective gloves/protective clothing/eye protection/face protection. Wash hands thoroughly after handling. Avoid contact with eyes. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Get medical advice/attention as soon as possible.

STORAGE

Keep tightly closed in dry well-ventilated place at 5-25°C. Protect from moisture, direct sunlight and heat sources. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

PACKAGING

REF 16300 20g powder, 10g liquid, measuring scoop, mixing pad

Made in EU for Dline OU
www.dline.ee



UAB „Medicinos linija“
Karaliaučiaus str. 29
LT-78348 Siauliai
Lithuania

Last revised: 2020-02