


dline Light Curing Temporary Filling Material

Light curing material for temporary filling and sealing of cavities

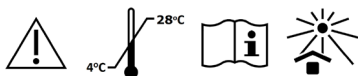

INSTRUCTION FOR USE EN

DESCRIPTION

dline Light Curing Temporary Filling Material is material for temporary use based on resins and inorganic filler. dline Light Curing Temporary Filling Material has easily packable consistency. This material is ideal for temporary fillings of short and extended periods.

INSTRUCTION FOR USE

NOTE! Do not use if primary package is broken.

1. Using suitable instrument take out a little of the material from the syringe and immediately close syringe cap.
2. Form a ball and place direct into cavity and adapt as necessary.
3. Do not apply layers more than 2 mm thickness.
4. Light curing for 20-40 seconds depending on layer thickness.

NOTE! After the desired amount of material extruded, the syringe cap must be closed immediately, so that the material is not unlighted.

WARNINGS AND PRECAUTIONS!

NOTE! Product contains urethane acrylate. Uncured product may cause sensitization by skin contact, may cause allergic reactions. Do not use dline Light Curing Temporary Filling Material if patient is allergic to product or to any of the ingredients.

Hazard and precautionary statements:

Product causes skin, serious eye irritation, may cause respiratory irritation. IF ON SKIN: Wash with plenty of soap and water. If skin irritation occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

Prevention:

Wash hands thoroughly after handling. Use only in a well-ventilated area. Wear protective gloves/protective clothing/eye protection/face protection.

Interaction with other substances:

dline Light Curing Temporary Filling Material should not be used with products containing eugenol.

STORAGE

Keep container tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze the product. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

PACKAGING

REF 19300 2x4g syringe

Made in EU for Dline OU
www.dline.ee



UAB "Medicinos linija"
Karaliaucius str. 29
LT-78348 Siauliai
Lithuania

Last revised: 2019-12