

dline Phosphoric Acid Etching Gel



INSTRUCTION FOR USE

EN

INDICATIONS

- Tooth enamel and / or dentin etching.

DESCRIPTION

dline Phosphoric Acid Etching Gel is 37% thixotropic phosphoric acid gel with optimum level of viscosity. Syringe presentation with special tips allows placement of the etching gel in any desired location.

INSTRUCTION FOR USE

NOTE! Do not use if primary package is broken. It is recommended to use cofferdam during application of the product.

PREPARATION:

1. Prepare surface as always. The working area must be kept dry.
2. If the surface to be etched is not freshly exposed, cleaning with oil-free prophylaxis paste such as dline Prophy Paste is recommended.
3. Attach tip applicator to the syringe.
4. Flow small amount of gel on mixing pad.

ETCHING:

1. Delicate push on plunger and apply layer of etch on all surface to be etched. Do not force plunger.
2. Leave etch gel in place for 15 seconds (dentine), 30 seconds (enamel).
3. Rinse with water and dry with air.
4. Verify etched surface is clean and dry.
5. Use adhesive such as dline Light Curing Adhesive.

WARNINGS AND PRECAUTIONS!

NOTE! Do not use dline Phosphoric Acid Etching Gel if patient is allergic to product or to any of the ingredients.

Hazard and precautionary statements:

dline Phosphoric Acid Etching Gel contains phosphoric acid, which causes skin corrosion, severe skin burns and eye damage. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. Wash contaminated clothing before reuse. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. Subsequently, have the casualty drink 1 - 2 glasses of water immediately. Do not try to neutralize with bases, no charcoal. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician.

Prevention:

Wash hands thoroughly after handling. Wear protective gloves / protective clothing / eye protection / face protection.

STORAGE

Keep tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

PACKAGING

REF 20040	4.3g syringe, 5 tips
REF 20120	12g syringe, 10 tips
REF 20450	42g syringe, 2x4.3g syringes, 20 tips
REF 20043	4x4.3g syringes, 20 tips

Made in EU for Dline OU
www.dline.ee



UAB „Medicinos linija“
Karaliaucius str. 29
LT-78348 Siauliai
Lithuania

Last revised: 2019-05