




Prophy Paste

Teeth surface polishing paste for tooth cleaning and polishing

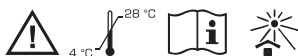

INSTRUCTION FOR USE

EN

DESCRIPTION

dline Prophy Paste is oil-free prophy paste.

Prophy paste is intended for use by dental professionals during professionally administered dental prophylaxis treatment:

- to clean debris, plaque, calculus, stains, and other accretions from the teeth;
- for the teeth surface preparation (cleaning and polishing) prior to restorations and fluoride treatment;
- for recontouring, finishing and polishing the surface after restoration to restore normal contours, smooth margins, and to create a high luster shine, the overall appearance and longevity of the restoration itself.

After evaluation of the teeth surface state and clinical situation dental professional may choose in one of several available levels of grit coarseness – fine (polishing), medium (polishing and cleaning) and coarse (cleaning).

DIRECTION FOR USE

NOTE! Do not use if primary package is broken.

1. Using suitable instrument take out a little of the paste and fill the prophy cup.
2. Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating of the tooth surface.
3. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.

WARNINGS AND PRECAUTIONS!

NOTE! Do not use dline Prophy Paste if patient is allergic to product or to any of the ingredients.

Prevention:

Wash hands thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection.

STORAGE

Keep container tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze the product. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

PACKAGING

REF	22000	30g paste coarse
REF	22010	30g paste medium
REF	22020	30g paste fine
REF	22001	100g paste coarse
REF	22011	100g paste medium
REF	22004	100x2g paste coarse
REF	22014	100x2g paste medium
REF	22024	100x2g paste fine

Made in EU for Dline OU
www.dline.ee