


Dline Root Canal Preparation Solution

**Root canal preparation solution to clean, softening and widening
the dentinal walls of the root canals**

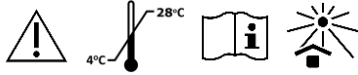

INSTRUCTION FOR USE **EN**

DESCRIPTION

dline Root Canal Preparation Solution is a root canal preparation solution based on EDTA acid.

DIRECTION FOR USE

NOTE! Do not use if primary package is broken. It is recommended to use cofferdam during application of the product.

1. Fill any suitable syringe with dline Root Canal Preparation Solution.
2. Attach irrigation needle onto syringe.
3. Delicate push the plunger and fill solution into canal.
4. Use root canal files to enlarge canal.
5. Repeat procedure until sufficient result is reached.
6. An alternating rinsing with 2.5 - 5% sodium hypochlorite solution such as dline Sodium Hypochlorite is recommended.

WARNINGS AND PRECAUTIONS!

NOTE! dline Root Canal Preparation Solution contains EDTA acid. Do not use product if patient is allergic to product or to any of the ingredients.

Hazard and precautionary statements:

IF ON SKIN: Wash with plenty of soap and water. If skin irritation occurs: Get medical advice/attention. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

Prevention:

Wash hands thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection.

STORAGE

Keep bottle tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze the product. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Do not use after the expiry date. Dispose of contents/bottle to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

PACKAGING

- | | |
|------------------|---------------------------------|
| REF 18250 | 50ml bottle |
| REF 18251 | 250ml bottle, 3ml empty syringe |

Made in EU for Dline OU
www.dline.ee



UAB „Medicinos linija“
Karaliaucius str. 29
LT-78348 Siauliai
Lithuania

Last revised: 2020-02