


Temporary Filling Material

For temporary filling and sealing of cavities and temporary fixing of post crowns

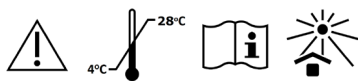

INSTRUCTION FOR USE

EN

DESCRIPTION

dline Temporary Filling Material is a self-curing, radiopaque, eugenol-free filling for temporary use based on zinc oxide and synthetic resins. dline Temporary Filling Material has easily packable consistency, high strength and hardness.

INSTRUCTION FOR USE

NOTE! Do not use if primary package is broken.

1. Using suitable instrument take out a little of the material, form a ball with your fingers and place direct into cavity. Adapt as necessary.
2. Material hardens by absorption of moisture from mouth.
3. For provisional fixing of post crowns spread a layer on inside of crowns and seal in normal way.
4. It is recommended that masticate pressure is not applied for about 1 hour.

WARNINGS AND PRECAUTIONS!

NOTE! Do not use dline Temporary Filling Material if patient is allergic to product or to any of the ingredients.

Prevention:

Wash hands thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection.

STORAGE

Keep container tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze the product. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

PACKAGING

REF 19102	38g paste, white
REF 19103	38g paste, pink

Made in EU for Dline OU
www.dline.ee



UAB „Medicinos linija“
Karaliaucius str. 29
LT-78348 Siauliai
Lithuania

Last revised: 2020-02