Prophy Paste

EN	Teeth surface polishing paste
LT	Dantų paviršiaus poliravimo pasta
DE	Polierpaste für die Zahnoberfläche
BG	Паста за полиране на повърхността на зъбите
CS	Pasta na leštění zubů
DA	Poleringspasta til tandoverflade
ET	Hambapinna poleerimis pasta
FR	Pâte pour le polissage des surfaces dentaires
EL	Πάστα στίλβωσης επιφάνειας δοντιών
HU	Fogak felületi polírozó paszta
IT	Pasta per lucidare la superficie dei denti
LV	Zobu virsmas pulēšanas pasta
NO	Tannpoleringstannkrem
PL	Pasta do polerowania powierzchni zębów
PT	Pasta de polimento de superfície de dentes
RO	Pastă pentru lustruirea suprafeței dentare
SK	Pasta na leštenie povrchu zubov
SL	Pasta za poliranje površine zob
ES	Pasta para pulir la superficie de los dientes
SV	Poleringspasta för tandytör
TR	Diş yüzeyi polisaj macunu
RU	Паста для полировки поверхности зубов
UA	Паста для полірування поверхні зубів

INSTRUCTION FOR USE

EN

DESCRIPTION

dline Prophy Paste is oil-free prophy paste for tooth cleaning and polishing. Dental professional may choose in one of several available levels of grit coarseness – fine (polishing), medium (polishing and cleaning) and coarse (cleaning).

COMPOSITION

Thickener 30-50%, pumice 20-40%, filler 10-30%, water 5-15%, flavouring agent 1-5 %, surfactant <1%, colorant <1%.

dline Prophy Paste does not contain medicinal substance, including human blood or plasma derivate; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

INTENDED PURPOSE AND CLINICAL BENEFITS

Prophy paste is intended for use by dental professionals during professionally administered dental prophylaxis treatment.

CLINICAL INDICATIONS

- to clean debris, plaque, calculus, stains, and other accretions from the teeth;
- for the teeth surface preparation (cleaning and polishing) prior to restorations and fluoride treatment;
- for recontouring, finishing and polishing the surface after restoration to restore normal contours, smooth margins, and to create a high luster shine, the overall appearance and longevity of the restoration itself.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

None known.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, product may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract irritation). Do not use product if patient is allergic to product or to any of the ingredients.

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth, saliva, mucosa.

INTENDED USER

DLINE PROPHY PASTE is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental materials. There is no need for specific training.

STERILITY

DLINE PROPHY PASTE is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

DLINE PROPHY PASTE is designed to be used in dental office where ambient temperature is 18-25°C. Do not re-use.

CONSUMABLE COMPONENTS AND ACCESSORIES

No consumable components and accessories are supplied with the device.

INSTRUCTION FOR USE

NOTE! Do not use if primary package is broken.

1. Using suitable instrument take out a little of the paste and fill the prophy cup.
2. Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating of the tooth surface.
3. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.

WARNINGS

Do not use dline Prophy Paste for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Prophy Paste does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS!

It is recommended to use cofferdam during application of the product. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. **IF ON SKIN OR MUCOSA:** Wash with plenty of soap and water. If skin irritation occurs: Get medical advice/attention. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing. Wash hands thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection.

Precautions to be taken in the event of changes in the performance of the device:

If noticed any discoloration of the patient's tooth tissue during use, strong pungent odor, uneven/non-homogenous consistency, or patient suffers from sudden acute pain on application site, or if any other abnormal behavior of the product noticed while manipulating the device, that was not mentioned above, discontinue to use immediately. Rinse applied product with plenty of water, use suction system and do not let the product enter the mouth and/or to be swallowed. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center. Collect all available remaining supplies, do not use them again and keep them out of reach in a safe place until further notice. Contact the manufacturer immediately and report of any noticed changes in the performance of the product.

SHELF-LIFE

Shelf-life of dline Prophy Paste is 3 years from the date of manufacture. Do not use after the expiry date. See packing for expiry date. Do not re-use.

STORAGE

Keep product tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

dline Prophy Paste is safe and performs as intended if it is used in accordance to manufacturer's instruction for use.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

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