Light Curing Glass Ionomer Filling Cement

EN	Light curing glass ionomer filling cement
LT	Šviesoje kietėjantis stiklo jonomerinis cementas plombavimui
DE	Lichthärtender Glasionomer-Füllzement
BG	Леко втвърдяващ се цемент от стъклоиономер
CS	Světlem tuhnoucí skloionomerní výplňový cement
DA	Let hærdende glasionomerfyldningscement
ET	Kergelt kõvenev klaasist ionomeeri täite tsement
FR	Ciment de remplissage verre ionomère photopolymérisable
EL	Ταιμνέτο γεμίματος ιοντομερούς από γυαλί ελαφριάς ακλήρυνσης
HU	Fénykeményedő üvegiómer töltőcement
IT	Cemento vetroionomerico fotopolimerizzabile per riempimento
LV	Gaismas cietēšanas stikla jonomēru pildījuma cements
NO	Lysherdende glasionomer fyllesement
PL	Światłoutwardzalny cement glasionomerowy wypełniający
PT	Cimento de enchimento de ionómero de vidro fotopolimerizável
RO	Ciment de umplere cu ionomer de sticlă fotopolimerizabil
SK	Svetlom tuhnutí skloionomerný výplňový cement
SL	Stekleno -ionomerni polnilni cement z lahkim strjevanjem
ES	Cemento de relleno de ionómero de vidrio fotopolimerizable
SV	Ljushärdande glasjonomerfyllningscement
TR	İşikla sertleşen cam iyonomer dolgu simanı
RU	Светоотверждаемый стеклоиономерный пломбирочный цемент
UA	Легкотвердіючий склоіономерний заливний цемент

EN ISO 9917-2 4.2c, Class 3

INSTRUCTION FOR USE EN

DESCRIPTION

dline Light Curing Glass Ionomer Filling Cement is light curing, bioinert, resin reinforced, glass ionomer filling cement. The material is easily mixable, radiopaque, with very high compressive, flexural and shear bond strengths. HEMA free formulation allows decrease allergic reactions. Available in Vita* shades.

COMPOSITION

Fluoroaluminosilicate glass 70-80%, water 5-10%, polyacrylic acid 5-10%, acids 5-10%, monomer 1-5%, inhibitors <2%, excipients <2%, photoinitiator <1%, pigment <1%.

dline Light Curing Glass Ionomer Filling Cement does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

flexural strength	54.2 MPa
-------------------	----------

INTENDED PURPOSE AND CLINICAL BENEFITS

dline Light Curing Glass Ionomer Filling Cement restores/improves aesthetic appearance of restorable tooth; restores/maintains dental function of restorable tooth; protects biological structures of restorable tooth and adjacent tissues.

CLINICAL INDICATIONS

- **For deciduous teeth fillings; restoration of class V/cervical lesions and small single surface class I and II cavities in adult teeth; glass ionomer/composite "sandwich" restorations; atraumatic restorative treatment or minimally invasive procedures; situations where cariostatic activity is required.**

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

dline Light Curing Glass Ionomer Filling Cement should not be used with products containing eugenol as eugenol may disturb polymerization process.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, dline Light Curing Glass Ionomer Filling Cement may cause allergic or irritation reactions (skin, eye, mucosa).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth, saliva.

INTENDED USER

dline Light Curing Glass Ionomer Filling Cement is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental cements. There is no need for specific training.

STERILITY

dline Light Curing Glass Ionomer Filling Cement is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular

maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

dline Light Curing Glass Ionomer Filling Cement is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of cement is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumables, such as measuring scoop, mixing pads, are supplied with the device.

INSTRUCTION FOR USE

1. Prepare cavity as always. For deep cavities use calcium hydroxide liners. For the best chemical bonding to the teeth, apply conditioner, such as dline Conditioner For Glass Ionomer Cements on prepared surface for 10 sec. Wash with water and dry with air.
2. Mixing ratio: 1 level spoon of powder / 1 drop of liquid. Lightly tap the bottle of powder against the hand before dispensing. Do not shake or invert. Hold liquid bottle vertically upright and squeeze gently when dispense a liquid.
3. For mixing use metal or plastic spatula and glass slab or paper mixing pad. Place powder and liquid on the pad and mix thoroughly with mixing spatula until exact consistency is reached. Mixing time 20 seconds.
4. The working area must be kept dry. Apply the mixed cement with suitable instrument into cavity and light cure for 20 seconds. Use LED polymerization lamp with light intensity 1200mW/cm² in full mode, not ramp or pulse mode. Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use. Total working time is 2 minutes including mixing.
5. After 7 minutes from light cure, the material is ready for finishing. Polishing stones and discs are recommended for finishing. Prevent excess heat generation.

WARNINGS

As dline Light Curing Glass Ionomer Filling Cement highly adhesive, clean all instruments before cement set on. Higher amount of powder or higher temperature shorten the working/setting time and lower temperature, or higher amount of liquid prolong the working/setting time. Do not use dline Light Curing Glass Ionomer Filling Cement for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Light Curing Glass Ionomer Filling Cement does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS!

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse.

Wash hands thoroughly after handling. Avoid contact with eyes. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

SHELF-LIFE

Shelf-life of dline Light Curing Glass Ionomer Filling Cement is 4 years of powder / 3 years of liquid from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 5-25°C. Protect from moisture, direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

dline Light Curing Glass Ionomer Filling Cement is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance will be introduced in EUDAMED as soon as it will start work.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

PACKAGING

REF 16120	10g powder A2, 4g liquid, measuring scoop, mixing pad
REF 16121	5g powder A2, 3g liquid, measuring scoop, mixing pad
REF 16130	10g powder A3, 4g liquid, measuring scoop, mixing pad
REF 16131	5g powder A3, 3g liquid, measuring scoop, mixing pad

* Registered trademark of the Vita Zahnfabrik H.Rauter GmbH & Co. KG, Bad Sackingen, Germany.

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