Line Conditioner For Glass Ionomer Cements

EN	Conditioner for glass ionomer cements
LT	Kondicionierius stiklo jonomeriniams cementams
DE	Konditionierer für Glasionomerzement
BG	Балсам за стъклоиономерни цименти
CS	Kondicionér na skloionomerní cementy
DA	Balsam til glasionomer -cement
ET	Palsam klaasionomeertsementidele
FR	Conditionneur pour ciments verre ionomère
EL	Κλιματιστικό για τσιμέντα γυάλινων ιονομερών
HU	Kondicionáló üvegiómer cementhez
IT	Condizionatore per cementi vetroionomerici
LV	Stikla jonomēru cementu kondicionieris
NO	Balsam for glassionomer sement
PL	Odżywka do cementów glasionomerowych
PT	Condicionador para cimentos de ionómero de vidro
RO	Balsam pentru cimenturi ionomer de sticlă
SK	Kondicionér na skloionomerné cementy
SL	Balzam za stekloionomerne cemente
ES	Acondicionador para cementos de ionómero de vidrio
SV	Balsam för glasionomer
TR	Cam ionomer simanları için saç kremi
RU	КОНДИЦИОНЕР для стеклоиономерных цементов
UA	КОНДИЦИОНЕР для склоіономерних цементів

INSTRUCTION FOR USE EN

DESCRIPTION

dline Conditioner For Glass Ionomer Cements is a tooth surface conditioner prior using glass ionomer cements.

COMPOSITION

Polyacrylic acid 60-80%, water 20-40%.
dline Conditioner For Glass Ionomer Cements does not contain medicinal substance, including human blood or plasma derivate; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

appearance	transparent liquid
------------	--------------------

INTENDED PURPOSE AND CLINICAL BENEFITS

dline Conditioner For Glass Ionomer Cements is intended to be used for tooth surface preparation prior glass ionomer cements.

CLINICAL INDICATIONS

- To condition the surface of the tooth prior glass ionomer cements.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

None known.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, dline Conditioner For Glass Ionomer Cements may cause allergic or irritation reactions (eye).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OR BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth.

INTENDED USER

dline Conditioner For Glass Ionomer Cements is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental materials. There is no need for specific training.

STERILITY

dline Conditioner For Glass Ionomer Cements is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

dline Conditioner For Glass Ionomer Cements is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of cement is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No consumable components or accessories are supplied with the device.

INSTRUCTION FOR USE

1. Prepare surface as always. For deep cavities use calcium hydroxide liner.
2. Apply dline Conditioner For Glass Ionomer Cements on prepared surface for 10 sec.
3. Wash with water and dry with air.

4. Apply glass ionomer cement, such as dline Glass Ionomer Filling Cement, dline Light Curing Glass Ionomer Filling Cement, dline Silver-Glass Ionomer Cement and finish restoration.

WARNINGS

Do not use dline Conditioner For Glass Ionomer Cements for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Conditioner For Glass Ionomer Cements does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

Wash hands thoroughly after handling. Avoid contact with eyes. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

SHELF-LIFE

Shelf-life of dline Conditioner For Glass Ionomer Cements is 5 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 5-25°C. Protect from moisture, direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

dline Conditioner For Glass Ionomer Cements is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance will be introduced in EUDAMED as soon as it will start work.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

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