Cline

# Dual Curing Core Build-Up And Post Cementation Material

EN	Dual curing core build-up and post cementation material
LT	Dvigubo kietėjimo kulties atstatymo ir kaiščių cementavimo medžiaga
DE	Dualhärtendes Stumpfaufbau- und Stiftbefestigungsmaterial
BG	Материал за двойно втвърдяване на сърцевината и след циментиране
CS	Materiál pro duální vytvrzování jádra a pocementování
DA	Dobbelt hærdende kerneopbygning og eftercermenteringsmateriale
ET	Kahekõvenev südamik ja tsementeerimisjärgne materjal
FR	Matériaux de construction de noyau et de post-cémentation à double durcissement
EL	Υλικό συσσώρευσης διπλού πυρήνα και μετά από τομέντο
HU	Kettős kikeményedő mag felépítő és utó cementáló anyag
IT	Materiale per la ricostruzione del moncone e post cementazione a polimerizzazione duale
LV	Divkārša cietēšanas koduls un pēc cementēšanas materiāls
NO	Dobbel herdende kjerneoppbygging og ettersementeringsmateriale
PL	Podwójnie utwardzalny materiał do odbudowy zębów i po zacementowaniu
PT	Formação de núcleo de cura dupla e material de pós-cimentação
RO	Material de întărire dublă și material de post-cimentare
SK	Materiál na dvojté vytvrdzovanie jadra a dodatočne cementačný materiál
SL	Dvojno strjevalno jedro za gradnjo in material po cementaciji
ES	Material de curado dual para reconstrucción de muñones y post cementación
SV	Dubbelhärdande kärnupplägg och eftercermenteringsmaterial
TR	Çift kürleñen kor oluşumu ve simantasyon sonrası malzeme
RU	Материал двойного отверждения для наращивания и последующей цементации
UA	Матеріал для нарощування та постцементації подвійного затвердіння

## INSTRUCTION FOR USE

EN

### DESCRIPTION

dline Dual Curing Core Build-Up And Post Cementation Material is a dual curing, bioinert, radiopaque highly aesthetic material which ensures strong adhesion to tooth surface and long lasting restoration. dline Dual Curing Core Build-Up And Post Cementation Material is mixed automatically.

### COMPOSITION

Base: dental glass grinded based on silica and barium 50-70%, methacrylate mixture 30-50%, silicon dioxide 1-5%, initiator <1%, opacifier <1%, inhibitor <1%. Catalyst: dental glass grinded based on silica and barium 50-70%, methacrylate mixture 30-50%, silicon dioxide 1-5%, coinitiator <1%, stabilizer <1%, photoinitiator <1%, opacifier <1%, inhibitor <1%, pigment <1%.

dline Dual Curing Core Build-Up And Post Cementation Material does not contain medicinal substance, including human blood or plasma derivate; tissues or cells, or their derivates, of human origin; tissues or cells of animal origin, or their derivates, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

### PERFORMANCE CHARACTERISTICS

dual curing	light and chemical
flexural strength	92.3±4.8 MPa
water sorption	18.49±0.15 µg/mm <sup>3</sup>
water solubility	0.75±0.26 µg/mm <sup>3</sup>

### INTENDED PURPOSE AND CLINICAL BENEFITS

dline Dual Curing Core Build-Up And Post Cementation Material restores/improves aesthetic appearance of restorable tooth; restores/maintains dental function of restorable tooth; protects biological structures of restorable tooth and adjacent tissues.

### CLINICAL INDICATIONS

- For core build-up and post cementation.

### CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

### RESTRICTIONS TO COMBINATIONS

dline Dual Curing Core Build-Up And Post Cementation Material should not be used with products containing eugenol as eugenol may disturb polymerization process.

### UNDESIRABLE SIDE EFFECTS

In susceptible individuals, dline Dual Curing Core Build-Up And Post Cementation Material may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract irritation).

### RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

### PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

### INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body - mouth. Tissues or body fluids contacted by the device - tooth.

### INTENDED USER

dline Dual Curing Core Build-Up And Post Cementation Material is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental materials. There is no need for specific training.

### STERILITY

dline Dual Curing Core Build-Up And Post Cementation Material is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

### USE ENVIRONMENT

dline Dual Curing Core Build-Up And Post Cementation Material is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of material is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

### CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumables, such as automix tips, intraoral-root canal tips, endo tips, are supplied with the device.

### INSTRUCTION FOR USE

#### PREPARATION:

1. Prepare contact surface using standard technique. The surface must be thoroughly rinsed and dried. In areas close to pulp use calcium hydroxide liner or glass ionomer base lining cement.
2. Place a transparent matrix and fix it in place.
3. Apply layer of etch, such as dline Phosphoric Acid Etching Gel on all surface to be etched. Leave etch in place for 15 seconds (dentine), 30 seconds (enamel). Rinse with water and dry with air. Avoid over drying dentin. After drying, the surface must be chalky and do not contaminated prior to applying the adhesive. If the surface becomes contaminated with saliva, rinse and dry again and re-etch if necessary, follow manufacturer's instruction for use.
4. Apply a thin layer of adhesive, such as dline Light Curing Adhesive immediately onto etched surface, follow manufacturer's instruction for use.
5. The working area must be kept dry.
6. Remove dline Dual Curing Core Build-Up And Post Cementation Material syringe cap.
7. Promptly and carefully attach the automix tip to the syringe.
8. Test flow of material from automix tip before using.
9. Squeeze out a 1-2mm of material onto a mixing pad to equal the base paste and catalyst paste in the automix tip. Do not use the paste which has been squeezed out.

#### POST CEMENTATION AND CORE BUILD-UP:

1. Using automix syringe and appropriate tip, apply dline Dual Curing Core Build-Up And Post Cementation Material directly into cavity. Keep the tip immersed in the material while dispensing to avoid air bubbles.
2. Insert prepared post. Light cure for 10 seconds to fix the position of the post.
3. Continue to apply the material around the post and form a core. Working time is about 165 seconds at 23°C.
4. For easier manipulation of a large amount of material, it is recommended to build-up core incrementally and light cure each layer for 5 seconds.
5. As final setting light cure for 20-40 seconds from labial (buccal) and lingual sides. Use LED polymerization lamp with light intensity 1200mW/cm<sup>2</sup>. Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use.
6. Chemical curing is about 3-4 minutes at 37°C.

#### FINISH:

1. Leave mixing tip on the syringe and change the tip prior to the next use.
2. When material has hardened fully, remove a matrix if was used.
3. Finish using standard technique.

#### WARNINGS

Increased humidity and/or temperature will reduce the working/setting time. Vice versa decreased humidity and/or temperature will prolong the working/setting time. The device is sensitive to light. The working time significantly reduces during application under intensive lighting. When removing dline Dual Curing Core Build-Up And Post Cementation Material from refrigerator after storage, leave the material at ambient temperature for some time until warm-up. Do not use dline Dual Curing Core Build-Up And Post Cementation Material for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Dual Curing Core Build-Up And Post Cementation Material does not emit radiation and does not cause any electromagnetic interferences.

#### PRECAUTIONS!

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF SWALLOWED: Rinse mouth. Call a Poison Center or doctor/physician if you feel unwell. IF INHALED: Remove person to fresh air and keep comfortable for breathing.

Wash hands thoroughly after handling. Use only in a well-ventilated area. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

*Precautions to be taken in the event of changes in the performance of the device:*  
If, during the use of the product, noticed non-homogenous consistency, product does not harden or does not adhere on the applied surface while curing with light or any by-products/phases are released during curing, or sudden acute pain occur on application site, or if any other abnormal behavior of the product noticed while manipulating the device, that is not mentioned above, discontinue to use immediately. Remove the restoration from the tooth cavity with suitable dental instrument do not let the product to be swallowed. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center. Collect all available remaining supplies, do not use them again and keep them out of reach in a safe place until further notice. Contact the manufacturer immediately and report of any noticed changes in the performance of the product.

#### SHELF-LIFE

Shelf-life of dline Dual Curing Core Build-Up And Post Cementation Material is 2 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

#### STORAGE

Keep product tightly closed in dry well-ventilated place at 2-8°C (preferable in refrigerator). Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

#### **DISPOSAL**

Dispose of contents/container to as required by national regulatory requirements.

#### **VIGILANCE**

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

#### **SUMMARY OF SAFETY AND CLINICAL PERFORMANCE**

dline Dual Curing Core Build-Up And Post Cementation Material is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance is available through manufacturer's website [www.dline.ee/sscp/](http://www.dline.ee/sscp/) until European Database on Medical Devices (EUDAMED) comes online.

#### **MANUFACTURERS RESPONSIBILITY**

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

#### **VALIDITY**

Upon publication of this instruction for use, all previous versions are superseded.

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### CEMENTÁCIA KERAMICKÉJ FAZETY A TVORBA JADRA:

1. Using automix syringe and appropriate tip, apply dline Dual Curing Core Build-Up And Post Cementation Material directly into cavity. Keep the tip immersed in the material while dispensing to avoid air bubbles.
2. Insert prepared post. Light cure for 10 seconds to fix the position of the post.

3. Continue to apply the material around the post and form a core. Working time is about 165 seconds at 23°C.
4. For easier manipulation of a large amount of material, it is recommended to build-up core incrementally and light cure each layer for 5 seconds.
5. As final setting light cure for 20-40 seconds from labial (buccal) and lingual sides. Use LED polymerization lamp with light intensity 1200mW/cm<sup>2</sup>. Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use.
6. Chemical curing is about 3-4 minutes at 37°C.

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