# Glass Ionomer Base Lining Cement

<b>EN</b>	Glass ionomer base lining cement
<b>LT</b>	Stiklo jonomerinis cementas bazėms ir pamušalams
<b>DE</b>	Auskleidungszement auf Glasionomerbasis
<b>BG</b>	Цимент за облицовка на основата на стъкльономер
<b>CS</b>	Skloionomerní cementová vyzdívka
<b>DA</b>	Foring af cement med glasionomer
<b>ET</b>	Klaasionomeerist alusvoodri tsement
<b>FR</b>	Ciment de revêtement à base de verre ionomère
<b>EL</b>	Τσιμέντο επένδυσης από γυαλί ιονομερούς βάσης
<b>HU</b>	Üveg ionomer alap béles cement
<b>IT</b>	Cemento di rivestimento a base di vetroionomero
<b>LV</b>	Stikla jonomēru bāzes odeses cements
<b>NO</b>	Glassionomer base liner sement
<b>PL</b>	Cement bazowy glasionomerowy
<b>PT</b>	Cimento para revestimento de base de ionómero de vidro
<b>RO</b>	Ciment de căptușire a bazei ionomer de sticlă
<b>SK</b>	Skloionomerový cementový obkladový cement
<b>SL</b>	Cement za podlogo iz stekloionomera
<b>ES</b>	Cemento de revestimiento a base de ionómero de vidrio
<b>SV</b>	Glasionomer basfoder cement
<b>TR</b>	Cam iyonomer taban astar gimentosu
<b>RU</b>	Футеровочный цемент на основе стеклюномера
<b>UA</b>	Цемент з футеровкою на основі склюіономерів

EN ISO 9917-1 4.2b Glass Polyalkenoate Cement

## INSTRUCTION FOR USE EN

### DESCRIPTION

dline Glass Ionomer Base Lining Cement is a chemical curing, bioinert, fast-setting, radiopaque glass ionomer cement with excellent thermal expansion and high strength and hardness.

### COMPOSITION

Fluoroaluminosilicate glass 50-55%, water 15-20%, polyacrylic acid 15-20%, zinc oxide 10-15%, excipients 0-3%.

dline Glass Ionomer Base Lining Cement does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

### PERFORMANCE CHARACTERISTICS

compressive strength	106.9 MPa
----------------------	-----------

### INTENDED PURPOSE AND CLINICAL BENEFITS

dline Glass Ionomer Base Lining Cement restores/maintains dental function of restorable tooth; protects biological structures of restorable tooth and adjacent tissues.

### CLINICAL INDICATIONS

- For base linings under composite restorations.

### CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

### RESTRICTIONS TO COMBINATIONS

None known.

### UNDESIRABLE SIDE EFFECTS

In susceptible individuals, dline Glass Ionomer Base Lining Cement may cause allergic or irritation reactions (eye).

### RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

### PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

### INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body - mouth. Tissues or body fluids contacted by the device - tooth.

### INTENDED USER

dline Glass Ionomer Base Lining Cement is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental cements. There is no need for specific training.

### STERILITY

dline Glass Ionomer Base Lining Cement is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

### USE ENVIRONMENT

dline Glass Ionomer Base Lining Cement is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of cement is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

### CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumables, such as measuring scoop, mixing pads, are supplied with the device.

### INSTRUCTION FOR USE

1. Prepare surface as always. For deep cavities use calcium hydroxide liner.
2. Mixing ratio: 1 level spoon of powder / 1 drop of liquid. Lightly tap the bottle of powder against the hand before dispensing. Do not shake or invert. Hold liquid bottle vertically upright and squeeze gently when dispense a liquid.
3. For mixing use metal or plastic spatula and glass slab. Divide powder into two parts. Mix the first half of powder with liquid for 10 seconds, and then add remainder. Mix until exact consistency is reached. Total mixing time is 20 seconds.
4. The working area must be kept dry. Apply the mixed cement with suitable instrument into cavity. Total working time is 2 minute including mixing.
5. After 6 minutes from start of mixing lining is ready for continue restoration.

### WARNINGS

As dline Glass Ionomer Base Lining Cement is highly adhesive, clean all instruments before cement set on. Higher amount of powder or higher temperature shorten the working/setting time and lower temperature, or higher amount of liquid prolong the working/setting time. Do not use dline Glass Ionomer Base Lining Cement for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Glass Ionomer Base Lining Cement does not emit radiation and does not cause any electromagnetic interferences.

### PRECAUTIONS!

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

Wash hands thoroughly after handling. Avoid contact with eyes. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

### Precautions to be taken in the event of changes in the performance of the device:

If noticed any changes in the performance of the device, immediately remove applied cement with suitable dental instrument. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center.

### SHELF-LIFE

Shelf-life of dline Glass Ionomer Base Lining Cement is 3 years of powder / 5 years of liquid from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

### STORAGE

Keep product tightly closed in dry well-ventilated place at 5-25°C. Protect from moisture, direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

### DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

### VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

dline Glass Ionomer Base Lining Cement is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance will be introduced in EUDAMED as soon as it will start work.

### MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

### VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

### PACKAGING

**REF 16300** 20g powder, 10g liquid, measuring scoop, mixing pad

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