Glass Ionomer Filling Cement

EN	Glass ionomer filling cement
LT	Stiklo jonomerinis cementas plombavimui
DE	Glasionomer-Füllzement
BG	Съкло -йонимерен пълнеж цемент
CS	Skloionomerní výplňový cement
DA	Glasionomerfyldningscement
ET	Klaasionomeeri täite tsement
FR	Ciment de remplissage verre ionomère
EL	Τσιμέντο γέμισης ιονομερών υαλιού
HU	Üveg ionomer töltő cement
IT	Cemento vetroionomerico di riempimento
LV	Stikla jonomeŗu pildījuma cements
NO	Sement av glassionomer
PL	Cement glasjonomerowy wypełniający
PT	Cimento de enchimento de ionómero de vidro
RO	Ciment de umplere cu ionomer de sticlă
SK	Skloionomerný plniaci cement
SL	Stekloionomerni polnilni cement
ES	Cemento de relleno de ionómero de vidrio
SV	Glasionomerfyllningscement
TR	Cam iyonomer dolgu simanı
RU	Стеклоиономерный цемент для заливки
UA	Склоіономерний заливний цемент

EN ISO 9917-1 4.2c Glass Polyalkenoate Cement

INSTRUCTION FOR USE EN

DESCRIPTION

dline Glass Ionomer Filling Cement is a chemical curing, bioinert, fast-setting, aesthetic glass ionomer cement with excellent thermal expansion and translucency in Vita* shades.

COMPOSITION

Fluoroaluminosilicate glass 80-100%, water 10-15%, polyacrylic acid 0-16%, excipient 0-2%, pigment 0-1%.

dline Glass Ionomer Filling Cement does not contain medicinal substance, including human blood or plasma derivate; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

compressive strength	264.7 MPa
lead content	0.16 mg/kg

INTENDED PURPOSE AND CLINICAL BENEFITS

dline Glass Ionomer Filling Cement restores/improves aesthetic appearance of restorable tooth; restores/maintains dental function of restorable tooth; protects biological structures of restorable tooth and adjacent tissues.

CLINICAL INDICATIONS

- For restoration of class III and V cavities, pits and fissures, class I and II cavities in deciduous teeth.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

None known.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, dline Glass Ionomer Filling Cement may cause allergic or irritation reactions (eye).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OR BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth, oral mucosa, saliva.

INTENDED USER

dline Glass Ionomer Filling Cement is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental cements. There is no need for specific training.

STERILITY

dline Glass Ionomer Filling Cement is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

dline Glass Ionomer Filling Cement is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of cement is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumables, such as measuring scoop, mixing pads, are supplied with the device.

INSTRUCTION FOR USE

1. Prepare surface as always. For deep cavities use calcium hydroxide liners. For the best chemical bonding to the teeth, apply conditioner on prepared surface for 10 sec. Wash with water and dry with air.
2. Mixing ratio: 3 level spoons of powder / 2 drops of liquid. Lightly tap the bottle of powder against the hand before dispensing. Do not shake or invert. Hold liquid bottle vertically upright and squeeze gently when dispense a liquid.
3. For mixing use metal or plastic spatula and glass slab or paper mixing pad. Divide powder into two parts. Mix the first half of powder with liquid for 10 seconds, and then add remainder. Mix until exact consistency is reached. Total mixing time is 30 seconds. Do not add powder in small portions.
4. The working area must be kept dry. Apply the mixed cement with suitable instrument into cavity. Total working time is 2 minute including mixing.
5. After 8 minutes from start of mixing restoration is ready for finishing.

WARNINGS

As dline Glass Ionomer Filling Cement highly adhesive, clean all instruments before cement set on. Higher amount of powder or higher temperature shorten the working/setting time and lower temperature, or higher amount of liquid prolong the working/setting time. Do not use dline Glass Ionomer Filling Cement for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Glass Ionomer Filling Cement does not emit radiation and does not cause any electromagnetic interferences.

Precautions to be taken in the event of changes in the performance of the device:

If during the use of the product, after the components are well mixed, product consistency is non-homogenous; product does not harden or colour change is observed in a moment of curing, i.e. the cement does not correspond to the intended shade specified by the manufacturer or cement does not adhere to the applied surface while setting or any by-products/phases are released during the time of curing; or sudden acute pain occur on application site; or if any other abnormal behavior of the product is noticed while manipulating the device, that is not mentioned above, discontinue to use immediately. Remove the restoration from the tooth cavity with suitable dental instrument do not let the product enter the mouth and/or to be swallowed. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center. Collect all available remaining supplies, do not use them again and keep them out of reach in a safe place until further notice. Contact the manufacturer immediately and report of any noticed changes in the performance of the product.

PRECAUTIONS

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

Wash hands thoroughly after handling. Avoid contact with eyes. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

SHelf-LIFE

Shelf-life of dline Glass Ionomer Filling Cement is 6 years of powder / 5 years of liquid from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 5-25°C. Protect from moisture, direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

dline Glass Ionomer Filling Cement is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance is available through manufacturer's website www.dline.ee/sscp/ until European Database on Medical Devices (EUDAMED) comes online.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

PACKAGING

REF 16040	10g powder A2, 10g powder A3, 6g liquid, measuring scoop, mixing pad
REF 16020	2x10g powder A2, 6g liquid, measuring scoop, mixing pad
REF 16030	2x10g powder A3, 6g liquid, measuring scoop, mixing pad

* Registered trademark of the Vita Zahnfabrik H.Rauter GmbH & Co. KG, Bad Sackingen, Germany.

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