# dline Light Curing Compomer Liner

<b>EN</b>	Light curing compomer liner
<b>LT</b>	Šviesoje kietėjantis kompomero pamušalas
<b>DE</b>	Lichthärtender Compomer-Liner
<b>BG</b>	Леко втвърдяваща се компомерна подложка
<b>CS</b>	Světlem tuhnoucí kompomerní vložka
<b>DA</b>	Let hærdende kompomerfor
<b>ET</b>	Valguskõvastav kompomeeri vooder
<b>FR</b>	Liner compomère photopolymérisable
<b>EL</b>	Ελαφριά επένδυση κομπόμερ
<b>HU</b>	Fényre kötő kompomér béles
<b>IT</b>	Liner in compomero fotopolimerizzabile
<b>LV</b>	Gaismas cietēšanas kompomere
<b>NO</b>	Letthærdende kompomerfor
<b>PL</b>	Kompomer światłoutwardzalny
<b>PT</b>	Forro de compomero fotopolimerizável
<b>RO</b>	Căptuşeală pentru întărire usoară
<b>SK</b>	Svetlom tuhnuća vložka z kompoméru
<b>SL</b>	Komponentna podloga za svetlo utrjevanje
<b>ES</b>	Revestimiento de compomero fotopolimerizable
<b>SV</b>	Lätthårdande kompomerfoder
<b>TR</b>	Işıkla sertleşen kompomér astar
<b>RU</b>	Светоотверждаемый компомерный лайнер
<b>UA</b>	Компомерний вкладиш із легким затвердінням

## INSTRUCTION FOR USE EN

**DESCRIPTION**

dline Light Curing Compomer Liner is light curing, bioinert, radiopaque compomer liner based on polyacid-modified composite resins and glass ionomer filler. Such composition allows compomer expansion partially compensate curing contraction of composite materials. Thanks to the glass ionomer content compomer doubles the adhesion to dentin. dline Light Curing Compomer Liner prevents crack formations, minimizes micro fractures and secondary caries. Material could be used both as a usual liner (coating the floor of the cavity) and to coat the entire cavity (stress-breaking function).

**COMPOSITION**

Dental glass reactive 50-70%, methacrylate mixture 20-30%, modifying agent 1-5%, photoinitiator <1%, cointiator <1%, opacifier <1%, stabilizer <1%, inhibitor <1%.

dline Light Curing Compomer Liner does not contain medicinal substance, including human blood or plasma derivate; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

**PERFORMANCE CHARACTERISTICS**

light curing	20s
depth of cure	2.51±0.11 mm
flexural strenght	34.3±2.8 MPa

**INTENDED PURPOSE AND CLINICAL BENEFITS**

dline Light Curing Compomer Liner helps to restore/improve aesthetic appearance of restorable tooth; restores/maintains dental function of restorable tooth; protects biological structures of restorable tooth and adjacent tissues.

**CLINICAL INDICATIONS**

- **For lining under polymer-based restorative materials, amalgams and porcelain restorations.**

**CONTRA-INDICATIONS**

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

**RESTRICTIONS TO COMBINATIONS**

dline Light Curing Compomer Liner should not be used with products containing eugenol as eugenol may disturb polymerization process.

**UNDESIRABLE SIDE EFFECTS**

In susceptible individuals, dline Light Curing Compomer Liner may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract irritation).

**RESIDUAL RISKS**

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

**PATIENT TARGET GROUP**

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

**INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS**

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth.

**INTENDED USER**

dline Light Curing Compomer Liner is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental liners. There is no need for specific training.

**STERILITY**

dline Light Curing Compomer Liner is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or

calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

**USE ENVIRONMENT**

dline Light Curing Compomer Liner is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of liner is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

**CONSUMABLE COMPONENTS AND ACCESSORIES**

No accessories are supplied with the device. Consumables, such as application tips, are supplied with the device.

- INSTRUCTION FOR USE**
1. Prepare cavity as always. Cleaning with oil-free prophylaxis paste, such as dline Proply Paste is recommended. Remove all cleaning residues with water spray and dry with air. Avoid contamination with saliva during applications.
  2. In deep cavities use calcium hydroxide liner.
  3. Apply layer of etch, such as dline Phosphoric Acid Etching Gel to all surface to be etched, follow manufacturer's instruction for use. Leave etch in place for 15 seconds (dentine), 30 seconds (enamel). Rinse surface with water and dry with air. Avoid over drying dentin.
  4. Apply a layer of adhesive, such as dline Light Curing Adhesive immediately onto etched surface, follow manufacturer's instruction for use. The tooth cavity must be kept dry.
  5. dline Light Curing Compomer Liner could be applied without adhesive. In this case, adhesion is similar to that obtained with usual cements (glass ionomer cement, phosphate cement).
  6. Using application tip apply layer of dline Light Curing Compomer Liner in a thickness of 1mm.
  7. Cure with light for 20 seconds at least. Use LED polymerization lamp with light intensity 1200mW/cm². Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use.
  8. Apply the layer of composite, such as dline Light Curing Zirconium Reinforced Composite, dline Light Curing Nano Hybrid Composite, dline Light Curing Nano Flowable Composite, follow manufacturer's instruction for use, and finish restoration.

**WARNINGS**

After the desired amount of material extruded, immediately remove application tip and close the syringe cap, so that the material is not unlighted. The material is sensitive to light. Avoid too long manipulation time under intensive lighting. Compomer adheres to metal instruments and should be removed with a paper towel before it has set. Place the lights as close as possible to the material. Use of a matrix is recommended. Do not use dline Light Curing Compomer Liner for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Light Curing Compomer Liner does not emit radiation and does not cause any electromagnetic interferences.

**PRECAUTIONS**

It is recommended to use cofferdam during application of the product. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. **IF ON SKIN OR MUCOSA:** Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. **IF SWALLOWED:** Rinse mouth. Call a Poison Center or doctor/physician if you feel unwell. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing. Wash hands thoroughly after handling. Use only in a well-ventilated area. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

**SHELF-LIFE**

Shelf-life of dline Light Curing Compomer Liner is 4 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

**STORAGE**

Keep product tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

**DISPOSAL**

Dispose of contents/container to as required by national regulatory requirements.

**VIGILANCE**

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**SUMMARY OF SAFETY AND CLINICAL PERFORMANCE**

dline Light Curing Compomer Liner is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance will be introduced in EUDAMED as soon as it will start work.

**MANUFACTURERS RESPONSIBILITY**

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

**VALIDITY**

Upon publication of this instruction for use, all previous versions are superseded.

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