# dline Light Curing Nano Hybrid Composite Based On Zirconium

<b>EN</b>	Light curing nano hybrid composite based on zirconium
<b>LT</b>	Šviesoje kietėjantis nano hibridinis kompozitas cirkonio pagrindu
<b>DE</b>	Lichthärtendes Nano-Hybrid-Komposit auf Zirkonbasis
<b>BG</b>	Лекото втвърдяващ се нано хибриден композит на базата на цирконий
<b>CS</b>	Světlem tuhnoucí nano hybridní kompozit na bázi zirkonia
<b>DA</b>	Let hærdende nano hybrid komposit baseret på zirkonium
<b>ET</b>	Kergelt kõvenev tsirkooniumil põhinev nanohüriidkomposiit
<b>FR</b>	Composite nano hybride photopolymérisable à base de zirconium
<b>EL</b>	Νάνο υβριδικό σύνθετο ελαφριάς σκλήρυνσης με βάση το ζirkόνιο
<b>HU</b>	Fényre kötő cirkónium alapú nano hibrid kompozit
<b>IT</b>	Composito nanoibrido fotopolimerizzabile a base di zirconio
<b>LV</b>	Gaisā cietējošs nano hibrīda kompozīts uz cirkonija bāzes
<b>NO</b>	Lysherdende nano -hybridkomposit basert på zirkonium
<b>PL</b>	Światłoutwardzalny kompozyt nanohybrydowy na bazie cyrkonu
<b>PT</b>	Compósito nano-híbrido fotopolimerizável com base em zircônio
<b>RO</b>	Compozit compus nano hibrid cu fotopolimerizare pe bază de zirconiu
<b>SK</b>	Svetlo tuhnučí nano hybridný kompozit na báze zirkónia
<b>SL</b>	Nano hibridni kompozit, polimeriziran v svetlobi, na osnovi cirkonija
<b>ES</b>	Composite nanohíbrido fotopolimerizable a base de circonio
<b>SV</b>	Lätthärdande nanohybridkomposit baserad på zirkonium
<b>TR</b>	Zirkonyum bazlı ışıkla sertleşen nano hibrit kompozit
<b>RU</b>	Светоотверждаемый наногибридный композит на основе циркония
<b>UA</b>	Світлотверднучий наногібридний композит на основі цирконію

## INSTRUCTION FOR USE EN

**DESCRIPTION**  
dline Light Curing Nano Hybrid Composite Based On Zirconium is light curing, bioinert, radiopaque nano hybrid composite in Vita\* shades with high aesthetics, excellent polishability, colour stability, high wear resistance and strength.

**COMPOSITION**  
Zirconium containing silica and barium dental glass grinded 70-80%, methacrylate mixture 30-40%, silicon dioxide 1-5%, coinitiator <1%, photoinitiator <1%, stabilizer <1%, inhibitor <1%, opacifier <1%, pigment <1%.  
dline Light Curing Nano Hybrid Composite Based On Zirconium does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

**PERFORMANCE CHARACTERISTICS**

light curing	20s
depth of cure	2.30±0.06 – 2.84±0.03 mm
flexural strength	113.4±2.9 MPa
water sorption	21.72±0.57 µg/mm <sup>3</sup>
water solubility	0.85±0.17 µg/mm <sup>3</sup>

**INTENDED PURPOSE AND CLINICAL BENEFITS**  
dline Light Curing Nano Hybrid Composite Based On Zirconium restores/improves aesthetic appearance of restorable tooth; restores/maintains dental function of restorable tooth; protects biological structures of restorable tooth and adjacent tissues.

**CLINICAL INDICATIONS**  
- **For direct anterior and posterior restorations in all cavity classes; indirect restorations such as inlays, onlays; cores.**

**CONTRA-INDICATIONS**  
Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

**RESTRICTIONS TO COMBINATIONS**  
dline Light Curing Nano Hybrid Composite Based On Zirconium should not be used with products containing eugenol as eugenol may disturb polymerization process.

**UNDESIRABLE SIDE EFFECTS**  
In susceptible individuals, dline Light Curing Nano Hybrid Composite Based On Zirconium may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract irritation).

**RESIDUAL RISKS**  
Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

**PATIENT TARGET GROUP**  
No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

**INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS**  
Part of the body – mouth. Tissues or body fluids contacted by the device – tooth, oral mucosa, saliva.

**INTENDED USER**

dline Light Curing Nano Hybrid Composite Based On Zirconium is developed for professional use in dentistry only. Its user only licensed doctor/technician who has knowledge how to use common dental composites. There is no need for specific training.

**STERILITY**  
dline Light Curing Nano Hybrid Composite Based On Zirconium is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

**USE ENVIRONMENT**  
dline Light Curing Nano Hybrid Composite Based On Zirconium is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of composite is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

**CONSUMABLE COMPONENTS AND ACCESSORIES**  
No consumable components and accessories are supplied with the device.

**INSTRUCTION FOR USE**  
**PREPARATION:**

1. Prepare cavity as always. Cleaning teeth with oil-free prophylaxis paste, such as dline Propy Paste is recommended.
2. Use the minimally invasively technique for preparation of the cavity. All enamel margins in the anterior region must be bevelled. Do not bevel the margins in the posterior region and avoid slice preparations. Rinse with water and dry with air.
3. For deep cavities use calcium hydroxide liners or glass ionomer base lining cement.
4. When filling cavities with approximal sections place a transparent matrix and fix it in place.
5. Apply layer of etch, such as dline Phosphoric Acid Etching Gel on all surface to be etched. Leave etch in place for 15 seconds (dentine), 30 seconds (enamel). Rinse with water and dry with air. Avoid over drying dentin. After drying, the surface must be chalky and do not contaminated prior to applying the adhesive. If the surface becomes contaminated with saliva, rinse and dry again and re-etch if necessary.
6. Apply a thin layer of adhesive, such as dline Light Curing Adhesive immediately onto etched surface and follow direction for use for adhesive. The working area must be kept dry.

**RESTORATION:**

1. Apply the layer of composite with suitable instrument into cavity. Do not apply layers more than 2mm deep.
2. Light cure for 20 seconds per layer. Use LED polymerization lamp with light intensity 1200mW/cm<sup>2</sup> in full mode, not ramp or pulse mode. Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use.
3. Due to the effect of the oxygen in the air, a thin smear layer of unpolymerized material remains on the surface of each layer. This bonds the layers chemically and must not be touched or contaminated with moisture.
4. Finish restoration.

**INDIRECT METHOD:**

1. Use the minimally invasively technique for preparation of the cavity. To prevent the material fracturing, it must be at least 1.5mm thick in the lateral and vertical aspects. All internal line and point angles must be rounded. Avoid slice preparations. Prepare a flat cervical shoulder - do not bevel it. Any unavoidable undercuts must be blocked out with glass ionomer base lining cement. Use slightly tapering diamonds with rounded tips for the preparation. For deep cavities use calcium hydroxide liners or glass ionomer base lining cement.
2. Take the impression and fabricate temporary restoration, using acrylic resin. For cementation of temporary restoration use only non-eugenol cements, such as dline Glass Ionomer Luting Cement.
3. Cast the impression with dental stone in the laboratory. Allow the model to set and pull off the impression. Block out the undercuts and apply an oil-free separating agent to the model. Build up the inlay layers on the model. Build up the approximal and deep occlusal sections first. Do not apply layers more than 2mm deep. Light cure for 20 seconds per layer. The finished inlay is then released from the die and cured fully for 8 minutes in curing oven. Trim the occlusal surface with fissure burs and polish to a high lustre with silicone polishers and diamond paste. Clean the inlay with liquid soap and water, rinse and dry with air.
4. Remove the temporary restoration and clean the cavity. Exert gentle pressure on the inlay to check for precision of fit. Do not use force. If necessary, trim the fitting surfaces to improve the fit.
5. The occlusion and articulation must not be checked when trying the inlay in as this would risk fracturing it.
6. Apply layer of etch, such as dline Phosphoric Acid Etching Gel on all surface to be etched. Leave etch in place for 15 seconds (dentine), 30 seconds (enamel). Rinse with water and dry with air. Avoid over drying dentin. After drying, the surface must be chalky and must not be contaminated prior to applying the adhesive.
7. Apply a thin layer of adhesive, such as dline Light Curing Adhesive immediately onto etched surface and follow direction for use for adhesive.
8. For cementation use any available dual curing luting composite. Apply the mixed material with suitable instrument on working area. Carefully press the inlay into position and remove excess of the material. To make certain that the inlay does not slip out of its proper position, press on it gently with a ball-end plugger until it has been light cured fully. Light curing for 20 seconds from each side. Remove the excess material with fine-grid diamonds and diamond finishing strips. Check the occlusion and adjust if necessary. Finish and polish.

**WARNINGS**  
After the desired amount of material extruded, screw syringe plunger anticlockwise by a half to full turn to release residual pressure inside the syringe and immediately close the syringe cap, so that the material is not unlighted. The material is sensitive to light. Avoid too long manipulation time under intensive lighting. Do not use dline Light Curing Nano Hybrid Composite Based On Zirconium for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Light Curing Nano Hybrid Composite Based On Zirconium does not emit radiation and does not cause any electromagnetic interferences.

**PRECAUTIONS**

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF SWALLOWED: Rinse mouth. Call a Poison Center or doctor/physician if you feel unwell. IF INHALED: Remove person to fresh air and keep comfortable for breathing.

Wash hands thoroughly after handling. Use only in a well-ventilated area. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

#### **Precautions to be taken in the event of changes in the performance of the device:**

If during the use of the product noticed non-homogenous, uneven consistency, product does not harden or composite colour change is observed in a moment of light curing, i.e. the composite does not correspond to the intended shade specified by the manufacturer or/and by-products/phases are released during curing, or sudden acute pain occur on application site, or if any other abnormal behavior of the product noticed while manipulating the device, that is not mentioned above, discontinue to use immediately. Remove the restoration from the tooth cavity with suitable dental instrument do not let the product to be swallowed. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center. Collect all available remaining supplies, do not use them again and keep them out of reach in a safe place until further notice. Contact the manufacturer immediately and report of any noticed changes in the performance of the product.

#### **SHELF-LIFE**

Shelf-life of dline Light Curing Nano Hybrid Composite Based On Zirconium is 3 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

#### **STORAGE**

Keep product tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

#### **DISPOSAL**

Dispose of contents/container to as required by national regulatory requirements.

#### **VIGILANCE**

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

#### **SUMMARY OF SAFETY AND CLINICAL PERFORMANCE**

dline Light Curing Nano Hybrid Composite Based On Zirconium is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance is available through manufacturer's website [www.dline.ee/sscp/](http://www.dline.ee/sscp/) until European Database on Medical Devices (EUDAMED) comes online.

#### **MANUFACTURERS RESPONSIBILITY**

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

#### **VALIDITY**

Upon publication of this instruction for use, all previous versions are superseded.

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\* Registered trademark of the Vita Zahnfabrik H.Rauter GmbH & Co. KG, Bad Sackingen, Germany.