Line Light Curing Pit And Fissure Sealant

EN	Light curing pit and fissure sealant
LT	Šviesioje kietėjantis duobelių ir vagelių silantas
DE	Lichthärtender Gruben- und Fissurenversiegler
BG	Лек втвърдяващ яма и уплътнител за пукнатини
CS	Lehce vytvrzující jáma a puklinový tmel
DA	Let hærdende grube og sprækkemasse
ET	Kergelt kõvenev kaev ja pragude hermeetik
FR	Scellant photopolymérisable pour puits et fissures
EL	Ελαφρύ λάκκο ακλήρυσης και σφραγιστικό σχισμής
HU	Fénykeményedő gödör és repedéstömítő
IT	Sigillante fotopolimerizzabile per fossette e fessure
LV	Viegli cietējoša bedre un plaisu hermētiķis
NO	Let hærdende grop og sprekkeforsegling
PL	Światłoutwardzalny uszczelniając do wgłębień i bruzd
PT	Selante de fissuras e fossas fotopolimerizáveis
RO	Etanș de întărire ușoară și etanșant pentru fisuri
SK	Svetlo tvrdnúca jáma a puklinový tmel
SL	Lahka polimerizacijska jama in tesnilna masa
ES	Sellador fotopolimerizable de fosas y fisuras
SV	Lätt hårdande grop och sprickor
TR	İşikla sertleşen çukur ve fissür örtücü
RU	Светоотверждаемый герметик для ямок и фиссур
UA	Льготвердіюча яма та герметик для тріщин

INSTRUCTION FOR USE EN

DESCRIPTION

dline Light Curing Pit And Fissure Sealant is white, bioinert, opaque, light curing, highly flowable sealant especially suitable for sealing small pits and fissures.

COMPOSITION

Dental glass grinded 30-50%, methacrylate mixture 20-50%, silicon dioxide 1-5%, coinitiator <1%, photoinitiator <1%, inhibitor <1%, stabilizer <1%, opacifier <1%. dline Light Curing Pit And Fissure Sealant does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

light curing	20 s
depth of cure	3.79±0.07 mm

INTENDED PURPOSE AND CLINICAL BENEFITS

dline Light Curing Pit And Fissure Sealant improves aesthetic appearance; maintains dental function; protects biological structures.

CLINICAL INDICATIONS

– For sealing small pits and fissures.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

dline Light Curing Pit And Fissure Sealant should not be used with products containing eugenol as eugenol may disturb polymerization process.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, dline Light Curing Pit And Fissure Sealant may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract irritation).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth, saliva.

INTENDED USER

dline Light Curing Pit And Fissure Sealant is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental sealants. There is no need for specific training.

STERILITY

dline Light Curing Pit And Fissure Sealant is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

dline Light Curing Pit And Fissure Sealant is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of sealant is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumables, such as application tips, are supplied with the device.

INSTRUCTION FOR USE

1. Prepare surface as always. Cleaning with oil-free prophylaxis paste, such as dline Propy Paste or pumice is recommended.
2. Apply layer of etch, such as dline Phosphoric Acid Etching Gel on all surface to be etched. Leave etch in place for 30 seconds. Rinse with water and dry with air.
3. The working area must be kept dry and not contaminated.
4. Apply a thin layer of dline Light Curing Pit And Fissure Sealant immediately onto etched surface: twist a disposable dispensing tip onto the syringe. Lightly pressing the plunger of the syringe, apply sealant onto mixing pad. Then apply onto surface using disposable micro applicator.
5. Light curing for 20 seconds. Use LED polymerization lamp with light intensity 1200mW/cm² in full mode, not ramp or pulse mode. Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use.
6. Inspect sealant for complete coverage and voids.
7. Wipe surface with cotton roll and check for hardness and retention.

WARNINGS

Sealant is highly flowable do not force syringe plunger. After the desired amount of material extruded, immediately remove application tip and close the syringe cap, so that the material is not unlighted. The material is sensitive to light. Avoid too long manipulation time under intensive lighting. Do not use dline Light Curing Pit And Fissure Sealant for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Light Curing Pit And Fissure Sealant does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS!

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF SWALLOWED: Rinse mouth. Call a Poison Center or doctor/physician if you feel unwell. IF INHALED: Remove person to fresh air and keep comfortable for breathing. Wash hands thoroughly after handling. Use only in a well-ventilated area. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

SHELF-LIFE

Shelf-life of dline Light Curing Pit And Fissure Sealant is 4 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

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