Light Curing Self Etching Universal Adhesive

EN	Light curing self etching universal adhesive
LT	Šviesioje kietėjantis savaimė išdinantis universalus surišėjas
DE	Lichthärtendes selbststättendes Universaladhäsiv
BG	Леко втвърдяващо се ецващо универсално лепило
CS	Světlem tuhnoucí samoleptací univerzální lepidlo
DA	Lyshærdende selvetsende universallim
ET	Kergesti kõvenev isesöövitav universaalne liim
FR	Adhésif universel auto-mordantant photopolymérisable
EL	Ελαφριά ακλήρυνση αυτοκόλλητη καθολική κόλλα
HU	Fénykeményedő önmáró univerzális ragasztó
IT	Adesivo universale fotopolimerizzabile automordenzante
LV	Gaisā cietējoša paškodējoša universāla līme
NO	Lysherdende selvetsende universallim
PL	Światłoutwardzalny samotrąwiający uniwersalny klej
PT	Adesivo universal autopolimerizável e fotopolimerizável
RO	Adeziv universal pentru autocuprare cu fotopolimerizare
SK	Svetlom tuhnuće samoleptacie univerzálne lepidlo
SL	Univerzalno lepilo za samo strjevanje, ki se strdi
ES	Adhesivo universal fotopolimerizable autograbante
SV	Lätthärdande självetsande universallim
TR	İşikla sertleşen, kendinden aşındırıcı universal yapıştırıcı
RU	Светоотверждаемый универсальный самопротравливающийся клей
UA	Легкотвердіючий самотравлення універсальний клей

INSTRUCTION FOR USE

EN

DESCRIPTION

dline Light Curing Self Etching Universal Adhesive is a light curing, bioinert, single component universal self-etching adhesive which incorporates aspects of etching and bonding in a single bottle.

COMPOSITION

Methacrylate mixture 40-60%, MDP monomer 15-25%, alcohol 10-25%, coinitiator ≤7%, water ≤5%, silicon dioxide ≤3%, photoinitiator ≤2%, inhibitors ≤1%.

dline Light Curing Self Etching Universal Adhesive does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

light curing	10 s
shear bond strength (enamel)	19.6±1.0 MPa
shear bond strength (dentin)	23.2±1.0 MPa

INTENDED PURPOSE AND CLINICAL BENEFITS

dline Light Curing Self Etching Universal Adhesive helps to restore/improve aesthetic appearance of restorable tooth; helps to restore/maintain dental function of restorable tooth; helps to protect biological structures of restorable tooth and adjacent tissues.

CLINICAL INDICATIONS

- To bond light or dual curing composites and compomers to tooth structure (dentin and enamel);
- To bond inlays, onlays, crowns, bridges, veneers in combination with a light or dual curing luting materials;
- To improve retention of a restoration.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

dline Light Curing Self Etching Universal Adhesive should not be used with products containing eugenol as eugenol may disturb polymerization process.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, dline Light Curing Self Etching Universal Adhesive may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth.

INTENDED USER

dline Light Curing Self Etching Universal Adhesive is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental adhesive. There is no need for specific training.

STERILITY

dline Light Curing Self Etching Universal Adhesive is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

dline Light Curing Self Etching Universal Adhesive is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of adhesive is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumables, such as micro applicators, cup for application, are supplied with the device.

INSTRUCTION FOR USE

1. Prepare cavity as always. Cleaning with oil-free prophylaxis paste, such as dline Proply Paste is recommended. For deep cavities use calcium hydroxide liner or glass ionomer base lining cement.
2. The surface must be not contaminated prior to applying the adhesive.
3. Shake the bottle well before each use.
4. Delicate push on bottle and dispense few drops of adhesive into dispensing cup. Do not force bottle.
5. Using a fully saturated micro applicator tip, apply dline Light Curing Self Etching Universal Adhesive on the prepared surface.
NOTE! Turn off or turn away the lamp from the working surface so the material does not harden.
6. Under light pressure rub the applied adhesive using the same micro applicator for 20 seconds.
7. Blow the surface with strong air force for 5 seconds to evaporate excess solvent. Use a vacuum aspirator to prevent the bonding liquid from scattering.
8. Curing with light for 10 seconds. Use LED polymerization lamp with light intensity 1200mW/cm². Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use.
9. Apply luting cement (indirect restorations), such as dline Dual Curing Adhesive Resin Cement or light curing restorative material (direct restorations), such as dline Light Curing Zirconium Reinforced Composite, dline Light Curing Nano Hybrid Composite, dline Light Curing Nano Flowable Composite and finish restoration.

WARNINGS

After the desired amount of adhesive extruded, immediately close the bottle cap. . dline Light Curing Self Etching Universal Adhesive is a highly flammable liquid. When removing dline Light Curing Self Etching Universal Adhesive from refrigerator after storage, leave the bottle at room temperature for some time (about 15 minutes) until warm-up. For non-cut enamel surfaces, pre-etching with phosphoric acid gel, such as dline Phosphoric Acid Etching Gel is always recommended. Do not use dline Light Curing Self Etching Universal Adhesive on exposed pulp. Do not use dline Light Curing Self Etching Universal Adhesive for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Light Curing Self Etching Universal Adhesive does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF SWALLOWED: Rinse mouth. Call a Poison Center or doctor/physician if you feel unwell. IF INHALED: Remove person to fresh air and keep comfortable for breathing.

Wash hands thoroughly after handling. Use only in a well-ventilated area. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

Precautions to be taken in the event of changes in the performance of the device:

If noticed any discoloration of the patient's tooth tissue during use, strong pungent odor, uneven/non-homogenous consistency, or patient suffers from sudden acute pain on application site, or if any other abnormal behavior of the product noticed while manipulating the device, that was not mentioned above, discontinue to use immediately. Rinse applied product with plenty of water, use suction system and do not let the product enter the mouth and/or to be swallowed. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center. Collect all available remaining supplies, do not use them again and keep them out of reach in a safe place until further notice. Contact the manufacturer immediately and report of any noticed changes in the performance of the product.

SHELF-LIFE

Shelf-life of dline Light Curing Self Etching Universal Adhesive is 2 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 2-8°C (preferable in a refrigerator). Protect from direct sunlight. Keep away from heat/sparks/open flames/hot surfaces. — No smoking. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

dline Light Curing Self Etching Universal Adhesive is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance is available through manufacturer's website www.dline.ee/sscp/ until European Database on Medical Devices (EUDAMED) comes online.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

PACKAGING

REF 14010 5ml bottle, 5pcs micro applicators, cup

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