dline Phosphoric Acid Etching Liquid

EN	Phosphoric acid etching liquid
LT	Fosforo rūgšties išdinimo skystis
DE	Ätzflüssigkeit mit Phosphorsäure
BG	Течност за офорт с фосфорна киселина
CS	Leptací kapalina na bázi kyseliny fosforečné
DA	Fosforsyre ætsende væske
ET	Fosforhappe söövitusvedelik
FR	Liquide de gravure à l'acide phosphorique
EL	Υγρό χάραξης φωσφορικού οξέος
HU	Foszforsav -marató folyadék
IT	Acido fosforico liquido per incisione
LV	Fosforskābes kodināšanas šķidrums
NO	Fosforsyre etsende væske
PL	Płyn do trawienia kwasem fosforowym
PT	Líquido de corrosão com ácido fosfórico
RO	Lichid de gravat cu acid fosforic
SK	Tekutina na leptanie kyseliny fosforečnou
SL	Tekočina za jedkanje fosforne kisline
ES	Líquido de grabado de ácido fosfórico
SV	Fosforsyra etsningsvätska
TR	Fosforik asit aşındırma sıvısı
RU	Жидкость для травления на основе фосфорной кислоты
UA	Рідина для травлення фосфорної кислоти

INSTRUCTION FOR USE EN

DESCRIPTION

dline Phosphoric Acid Etching Liquid is 37% thixotropic phosphoric acid liquid with optimum level of viscosity.

COMPOSITION

Diluent 60-80%, phosphoric acid 37%, thickener <1%, pigment <1%.
dline Phosphoric Acid Etching Liquid does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

pH	<1
----	----

INTENDED PURPOSE AND CLINICAL BENEFITS

dline Phosphoric Acid Etching Liquid is intended to be used for tooth surface preparation procedures.

CLINICAL INDICATIONS

- **Tooth enamel and / or dentin etching.**

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

None known.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, dline Phosphoric Acid Etching Liquid may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth.

INTENDED USER

dline Phosphoric Acid Etching Liquid is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental etchants. There is no need for specific training.

STERILITY

dline Phosphoric Acid Etching Liquid is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

dline Phosphoric Acid Etching Liquid is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of etchant is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No consumable components and accessories are supplied with the device.

INSTRUCTION FOR USE

1. Prepare surface as always. The working area must be kept dry.
2. If the surface to be etched is not freshly exposed, cleaning with oil-free prophylaxis paste such as dline Prophyl Paste is recommended.
3. Apply layer of etch on all surface to be etched.

4. Leave etch liquid in place for 15 seconds (dentine), 30 seconds (enamel).
5. Rinse with water and dry with air.
6. Verify etched surface is clean and dry.
7. Use adhesive such as dline Light Curing Adhesive.

WARNINGS

Do not use dline Phosphoric Acid Etching Liquid for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Phosphoric Acid Etching Liquid does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS

It is recommended to use cofferdam during application of the product. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. **IF ON SKIN OR MUCOSA:** Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. **IF SWALLOWED:** Rinse mouth. Do NOT induce vomiting. Subsequently, have the casualty drink 1 - 2 glasses of water immediately. Do not try to neutralize with bases, no charcoal. Call a Poison Center or doctor/physician if you feel unwell. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing.

Wash hands thoroughly after handling. Use only in a well-ventilated area. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

Precautions to be taken in the event of changes in the performance of the device:

If noticed any discoloration and/or increased gloss of the patient's tooth tissue during use, sudden acute pain, strong pungent odor, foaming, uneven/run-off application or any other abnormal behaviour of the product, while manipulating the device, that is not mentioned above, discontinue to use immediately. Rinse applied etching liquid with plenty of water, use suction system and do not let the product enter the mouth and/or to be swallowed. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center. Continue to collect all available remaining supplies, do not use them again and keep them out of reach in a safe place until further notice. Contact the manufacturer immediately and report of any noticed changes in the performance of the product.

SHELF-LIFE

Shelf-life of dline Phosphoric Acid Etching Liquid is 3 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

dline Phosphoric Acid Etching Liquid is safe and performs as intended if it is used in accordance to manufacturer's instruction for use.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

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