# Resin Modified Glass Ionomer Luting Cement

<b>EN</b>	Resin modified glass ionomer luting cement
<b>LT</b>	Derva modifikuotas stiklo jonomerinis cementas cementavimui
<b>DE</b>	Harzmodifizierter Glasionomer-Befestigungszement
<b>BG</b>	Смолаво -модифициран цемент от стъклоиономер
<b>CS</b>	Přyskyřičí modifikovaný skloionomerní tmel
<b>DA</b>	Harpiks modificeret glas ionomer luting cement
<b>ET</b>	Vaiguga modifitseeritud klaasist ionomeerist liimtsement
<b>FR</b>	Ciment de scellement verre ionomère modifié à la résine
<b>EL</b>	Τσιμέντο ιοντομερούς γυαλιού τροποποιημένο με ρητίνη
<b>HU</b>	Gyantával módosított üvegionomer tömítő cement
<b>IT</b>	Cemento vetroionomerico modificato con resina
<b>LV</b>	Ar sveļkiem modificēts stikla jonomeņu pārklājošais cements
<b>NO</b>	Harpiks modifisert glass ionomer luting sement
<b>PL</b>	Cement glasonomerowy modyfikowany żywicą
<b>PT</b>	Cimento de cimentação de ionômero de vidro modificado com resina
<b>RO</b>	Ciment ionic de sticlă modificat cu rășină
<b>SK</b>	Živcovo modifikovaný skloionomerný tmeľiaci cement
<b>SL</b>	Smola modifisirani cement za stekloionomerno cementiranje
<b>ES</b>	Cemento de fijación de ionómero de vidrio modificado con resina
<b>SV</b>	Harts -modifierad glasjonomer som luter cement
<b>TR</b>	Reçine modifiye cam iyonomer yapıştırma simanı
<b>RU</b>	Модифицированный смолой стеклоиономерный цемент для фиксации
<b>UA</b>	Цемент, що модифікується смолою з іономеру

EN ISO 9917-2 4.2a, Class 1

## INSTRUCTION FOR USE EN

### DESCRIPTION

dline Resin Modified Glass Ionomer Luting Cement is chemical curing, bioinert, radiopaque, resin modified glass ionomer luting cement with very high adhesion to dentin, enamel and alloys. Has low film thickness and good aesthetic. Available in one universal shade.

### COMPOSITION

Fluoroaluminosilicate glass 60-65%, water 10-15%, polyacrylic acid 10-15%, monomers 10-15%, acids <2%, inhibitors <2%, initiator <1%, pigment <1%.

dline Resin Modified Glass Ionomer Luting Cement does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

### PERFORMANCE CHARACTERISTICS

flexural strength	53.5 MPa
film thickness	14.2 µm

### INTENDED PURPOSE AND CLINICAL BENEFITS

dline Resin Modified Glass Ionomer Luting Cement restores/improves aesthetic appearance of restorable tooth; restores/maintains dental function of restorable tooth; protects biological structures of restorable tooth and adjacent tissues.

### CLINICAL INDICATIONS

- **For cementation of all metal, ceramic, porcelain and resin crowns, bridges, inlays, onlays; cementation of metal and ceramic orthodontic brackets.**

### CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

### RESTRICTIONS TO COMBINATIONS

dline Resin Modified Glass Ionomer Luting Cement should not be used with products containing eugenol as eugenol may disturb polymerization process.

### UNDESIRABLE SIDE EFFECTS

In susceptible individuals, dline Resin Modified Glass Ionomer Luting Cement may cause allergic or irritation reactions (skin, eye, mucosa).

### RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

### PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

### INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth, saliva.

### INTENDED USER

dline Resin Modified Glass Ionomer Luting Cement is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental cements. There is no need for specific training.

### STERILITY

dline Resin Modified Glass Ionomer Luting Cement is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

## USE ENVIRONMENT

dline Resin Modified Glass Ionomer Luting Cement is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of cement is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

## CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumables, such as measuring scoop, mixing pads, are supplied with the device.

## INSTRUCTION FOR USE

1. Prepare surface as always. For deep cavities use calcium hydroxide liner. Do not use eugenol-containing materials.
2. Mixing ratio: 1 level spoon of powder / 3 drops of liquid. Lightly tap the bottle of powder against the hand before dispensing. Do not shake or invert. Hold liquid bottle vertically upright and squeeze gently when dispense a liquid.
3. For mixing use metal or plastic spatula and glass slab or paper mixing pad. Place powder and liquid on the pad and mix thoroughly with mixing spatula until exact consistency is reached. Total mixing time is 30 seconds. Cement is pale pink in colour at this time.
4. The working area must be kept dry. Apply the mixed cement with suitable instrument to both restoration and prepared tooth immediately after mixing. Total working time is 2 minutes including mixing. Excess cement must be removed when cement is rubbery.
5. After 6 minutes from start of mixing restoration is ready for finishing.

## WARNINGS

As dline Resin Modified Glass Ionomer Luting Cement highly adhesive, clean all instruments before cement set on. Higher amount of powder or higher temperature shorten the working/setting time and lower temperature, or higher amount of liquid prolong the working/setting time. The colour of cement changes to a universal shade within 8 hours in the mouth. Do not use dline Resin Modified Glass Ionomer Luting Cement for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Resin Modified Glass Ionomer Luting Cement does not emit radiation and does not cause any electromagnetic interferences.

## PRECAUTIONS!

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse.

Wash hands thoroughly after handling. Avoid contact with eyes. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

## SHELF-LIFE

Shelf-life of dline Resin Modified Glass Ionomer Luting Cement is 2 years of powder / 2 years of liquid from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

## STORAGE

Keep product tightly closed in dry well-ventilated place at 5-25°C. Protect from moisture, direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

## DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

## VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

dline Resin Modified Glass Ionomer Luting Cement is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance will be introduced in EUDAMED as soon as it will start work.

## MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

## VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

## PACKAGING

<b>REF 16510</b>	7g powder, 4g liquid, measuring scoop, mixing pad
<b>REF 16500</b>	20g powder, 2x6.5g liquid, measuring scoop, mixing pad

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