Resin Modified Glass Ionomer Luting Cement

EN	Resin modified glass ionomer luting cement
LT	Derva modifikuotas stiklo jonomerinis cementas cementavimui
DE	Harzmodifizierter Glasionomer-Befestigungszement
BG	Смоново -модифициран цемент от стъклайонимер
CS	Přyskyřičí modifikovaný skloionomerní tmel
DA	Harpiks modificeret glas ionomer luting cement
ET	Vaiguga modifitseeritud klaasist ionomeerist liimtisement
FR	Ciment de scellement verre ionomère modifié à la résine
EL	Τσιμέντο ιοντομερούς υαλοίου τροποποιημένο με ρητίνη
HU	Gyantával módosított üvegiómoner tömítő cement
IT	Cemento vetroionomerico modificato con resina
LV	Ar sveķiem modificēts stikla jonomēru pārkļājošais cements
NO	Harpiks modificeret glass ionomer luting cement
PL	Cement glasionomerowy modyfikowany żywicą
PT	Cimento de cimentação de iónmero de vidro modificado com resina
RO	Ciment ionic de sticlă modificat cu rășină
SK	Živcovo modifikovaný skloionomerný tmelci cement
SL	Smola modificiran cement za stekloionomerno cementiranje
ES	Cemento de fijación de ionómero de vidrio modificado con resina
SV	Harts -modifierad glasionomer som luter cement
TR	Reçine modifiye cam ionomer yapıştırma simanı
RU	Модифицированный смолой стеклоиономерный цемент для фиксации
UA	Цемент, що модифікується смолою з іономеру

EN ISO 9917-2 4.2a, Class 1

INSTRUCTION FOR USE EN

DESCRIPTION
dline Resin Modified Glass Ionomer Luting Cement is chemical curing, bioinert, radiopaque, resin modified glass ionomer luting cement with very high adhesion to dentin, enamel and alloys. Has low film thickness and good aesthetic. Available in one universal shade.

COMPOSITION
Fluoroaluminosilicate glass 60-65%, water 10-15%, polyacrylic acid 10-15%, monomers 10-15%, acids <2%, inhibitors <2%, initiator <1%, pigment <1%.
dline Resin Modified Glass Ionomer Luting Cement does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

flexural strength	53.5 MPa
film thickness	14.2 µm

INTENDED PURPOSE AND CLINICAL BENEFITS
dline Resin Modified Glass Ionomer Luting Cement restores/improves aesthetic appearance of restorable tooth; restores/maintains dental function of restorable tooth; protects biological structures of restorable tooth and adjacent tissues.

CLINICAL INDICATIONS
- For cementation of all metal bridges, inlays, onlays; cementation of metal orthodontic brackets to tooth structure.

CONTRA-INDICATIONS
Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS
dline Resin Modified Glass Ionomer Luting Cement should not be used with products containing eugenol as eugenol may disturb polymerization process.

UNDESIRABLE SIDE EFFECTS
In susceptible individuals, dline Resin Modified Glass Ionomer Luting Cement may cause allergic or irritation reactions (skin, eye, mucosa).

RESIDUAL RISKS
Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP
No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS
Part of the body – mouth. Tissues or body fluids contacted by the device – tooth, oral mucosa, saliva.

INTENDED USER
dline Resin Modified Glass Ionomer Luting Cement is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental cements. There is no need for specific training.

STERILITY
dline Resin Modified Glass Ionomer Luting Cement is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

dline Resin Modified Glass Ionomer Luting Cement is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of cement is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES
No accessories are supplied with the device. Consumables, such as measuring scoop, mixing pads, are supplied with the device.

- INSTRUCTION FOR USE**
1. Prepare surface as always. For deep cavities use calcium hydroxide liner. Do not use eugenol-containing materials.
 2. Mixing ratio: 1 level spoon of powder / 3 drops of liquid. Lightly tap the bottle of powder against the hand before dispensing. Do not shake or invert. Hold liquid bottle vertically upright and squeeze gently when dispense a liquid.
 3. For mixing use metal or plastic spatula and glass slab or paper mixing pad. Place powder and liquid on the pad and mix thoroughly with mixing spatula until exact consistency is reached. Total mixing time is 30 seconds. Cement is pale pink in colour at this time.
 4. The working area must be kept dry. Apply the mixed cement with suitable instrument to both restoration and prepared tooth immediately after mixing. Total working time is 2 minutes including mixing. Excess cement must be removed when cement is rubbery.
 5. After 6 minutes from start of mixing restoration is ready for finishing.

WARNINGS
As dline Resin Modified Glass Ionomer Luting Cement highly adhesive, clean all instruments before cement set on. Higher amount of powder or higher temperature shorten the working/setting time and lower temperature, or higher amount of liquid prolong the working/setting time. The colour of cement changes to a universal shade within 8 hours in the mouth. Do not use dline Resin Modified Glass Ionomer Luting Cement for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Resin Modified Glass Ionomer Luting Cement does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS!
It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. Wash hands thoroughly after handling. Avoid contact with eyes. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

Precautions to be taken in the event of changes in the performance of the device:
If during the use of the product, after all the components are well mixed, product consistency is non-homogenous, or product does not harden or does not adhere to the applied surface while setting or any by-products/phases are released during the time of curing, or sudden acute pain occur on application site, or if any other abnormal behavior of the product is noticed while manipulating the device, that is not mentioned above, discontinue to use immediately. Remove the restoration from the tooth cavity with suitable dental instrument, do not let the product to be swallowed. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center. Collect all available remaining supplies, do not use them again and keep them out of reach in a safe place until further notice. Contact the manufacturer immediately and report of any noticed changes in the performance of the product.

SHELF-LIFE
Shelf-life of dline Resin Modified Glass Ionomer Luting Cement is 2 years of powder / 2 years of liquid from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE
Keep product tightly closed in dry well-ventilated place at 5-25°C. Protect from moisture, direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL
Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE
If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE
dline Resin Modified Glass Ionomer Luting Cement is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance is available through manufacturer's website www.dline.ee/sscp/ until European Database on Medical Devices (EUDAMED) comes online.

MANUFACTURERS RESPONSIBILITY
Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY
Upon publication of this instruction for use, all previous versions are superseded.

PACKAGING

REF 16510	7g powder, 4g liquid, measuring scoop, mixing pad
REF 16500	20g powder, 2x6.5g liquid, measuring scoop, mixing pad

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Commented [id/RV1]:
- Cementing of all metal bridges, inlays and onlays.
- Cementing of metal orthodontic brackets to tooth structure.

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