Root Canal Preparation Solution

EN	Root canal preparation solution
LT	Šaknies kanalo paruošimo tirpalas
DE	Lösung zur Wurzelkanalauflbereitung
BG	Разтвор за подготовка на кореновия канал
CS	Roztok pro přípravu kořenového kanálu
DA	Root kanal forberedelse løsning
ET	Juurekanali ettevalmistamise lahus
FR	Solution de préparation canalaire
EL	Διάλυμα παρασκευής ριζικού καναλιού
HU	Gyökércsatorna -előkészítő oldat
IT	Soluzione per la preparazione del canale radicolare
LV	Sakņu kanālu sagatavošanas šķīdums
NO	Forberedelse av rotkanal
PL	Roztwór do przygotowania kanału korzeniowego
PT	Solução de preparação de canal radicular
RO	Soluție de preparare a canalului radicular
SK	Roztok na prípravu koreňového kanálka
SL	Raztopina za pripravo koreninskega kanala
ES	Solución de preparación de conductos radiculares
SV	Rotkanalberedningslösning
TR	Kanal hazırlama solüsyonu
RU	Раствор для препарирования корневых каналов
UA	Розчин для підготовки корневих каналів

INSTRUCTION FOR USE EN

DESCRIPTION

dline Root Canal Preparation Solution is root canal preparation solution based on EDTA acid (17%).

COMPOSITION

Diluent 80-85%, Disodium EDTA 17%, buffering agent > 1%, emulsifying agent <1%.

dline Root Canal Preparation Solution does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

pH	7.83±0.01
----	-----------

INTENDED PURPOSE AND CLINICAL BENEFITS

dline Root Canal Preparation Solution is intended for chemomechanical root canal preparation procedures.

CLINICAL INDICATIONS

- Helps to clean, softening and widening the dentinal walls of the root canals.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

None known.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, dline Root Canal Preparation Solution may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth.

INTENDED USER

dline Root Canal Preparation Solution is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental solutions. There is no need for specific training.

STERILITY

dline Root Canal Preparation Solution is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

dline Root Canal Preparation Solution is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of solution is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumable, such as empty syringe, is supplied with the device.

INSTRUCTION FOR USE

1. Fill any suitable syringe with dline Root Canal Preparation Solution.
2. Attach irrigation needle onto syringe.
3. Delicate push the plunger and fill solution into canal.
4. Use root canal files to enlarge canal.
5. Repeat procedure until sufficient result is reached.
6. An alternating rinsing with 2.5 - 5% sodium hypochlorite solution is recommended.

WARNINGS

Do not use dline Root Canal Preparation Solution for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Root Canal Preparation Solution does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS!

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of soap and water. If skin irritation occurs: Get medical advice/attention. IF INHALED: Remove person to fresh air and keep comfortable for breathing. Wash hands thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection.

Precautions to be taken in the event of changes in the performance of the device:

If, during the use of the product, noticed non-homogenous product consistency, turbidity, precipitates, strong pungent odor, discoloration of the patient's tooth tissue, foaming, or patient experience sudden acute pain on the application site, or any other abnormal behavior or appearance of the product is noticed while manipulating the device, that is not mentioned above, discontinue to use immediately. Rinse applied product with plenty of water, use suction system and do not let the product enter the mouth and/or to be swallowed. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center. Collect all available remaining supplies, do not use them again and keep them out of reach in a safe place until further notice. Contact the manufacturer immediately and report of any noticed changes in the performance of the product.

SHELF-LIFE

Shelf-life of dline Root Canal Preparation Solution is 3 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

dline Root Canal Preparation Solution is safe and performs as intended if it is used in accordance to manufacturer's instruction for use.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

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