Temporary Filling Material

EN	Temporary filling material
LT	Laikinas užpildas
DE	Provisorisches Füllmaterial
BG	Временни материали за пълнене
CS	Dočasný výplňový materiál
DA	Midlertidigt påfyldningsmateriale
ET	Ajutine täitematerjal
FR	Matériau de remplissage temporaire
EL	Προσωρινό υλικό πλήρωσης
HU	Ideiglenes töltőanyag
IT	Materiale di riempimento temporaneo
LV	Pagaidu pildījuma materiāls
NO	Midlertidig fyllmateriale
PL	Tymczasowy materiał wypełniający
PT	Material de enchimento temporário
RO	Material de umplere temporar
SK	Dočasný výplňový materiál
SL	Začasni polnilni material
ES	Material de relleno temporal
SV	Tillfälligt fyllmaterial
TR	Gecici dolgu malzemesi
RU	Временный пломбирочный материал
UA	Тимчасовий пломбувальний матеріал

INSTRUCTION FOR USE EN

DESCRIPTION

dline Temporary Filling Material is a self-curing, eugenol-free, bioinert filling for temporary use based on zinc oxide and synthetic resins. dline Temporary Filling Material has easily packable consistency, excellent retention, and it is easy to use.

COMPOSITION

Zinc oxide 20-50%, calcium sulfate 20-50%, solvent 10-30%, filler 5-20%, polymer 5-20%, odour <1%, pigment <1%.

dline Temporary Filling Material does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

setting	in presence of moisture
---------	-------------------------

INTENDED PURPOSE AND CLINICAL BENEFITS

dline Temporary Filling Material restores/improves for temporary aesthetic appearance of restorable tooth; restores/maintains for temporary dental function of restorable tooth; protects for temporary biological structures of restorable tooth and adjacent tissues.

CLINICAL INDICATIONS

- For temporary fillings.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

None known.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, dline Temporary Filling Material may cause allergic or irritation reactions (skin, eye, mucosa).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth, oral mucosa, saliva.

INTENDED USER

dline Temporary Filling Material is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental materials. There is no need for specific training.

STERILITY

dline Temporary Filling Material is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

dline Temporary Filling Material is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of material is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No consumable components and accessories are supplied with the device.

INSTRUCTION FOR USE

- Using suitable instrument take out a little of the material, form a ball with your fingers and place direct into cavity. Adapt as necessary.
- Do not apply layers more than 2 mm thickness.
- Material hardens by absorption of moisture from mouth.
- It is recommended that masticate pressure is not applied for about 1 hour.
- Use rotary instruments to remove the temporary filling from oral cavity after no longer than 30 days.

WARNINGS

Do not use dline Temporary Filling Material for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. dline Temporary Filling Material does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS

It is recommended to use cofferdam during application of the product. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. **IF ON SKIN:** Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. Wash hands thoroughly after handling. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

Precautions to be taken in the event of changes in the performance of the device:

If, during the use of the product, noticed non-homogenous consistency, product colour or consistency has changed (the product is too stiff/watery, it is difficult to knead and shape it), if product does not harden in the oral cavity or does not adhere on the applied surface while hardening or emits any by-products/phases while curing, or sudden acute pain occur on application site, or if any other abnormal behavior of the product noticed while manipulating the device, that is not mentioned above, discontinue to use immediately. Remove the restoration from the tooth cavity with suitable dental instrument do not let the product to be swallowed. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center. Collect all available remaining supplies, do not use them again and keep them out of reach in a safe place until further notice. Contact the manufacturer immediately and report of any noticed changes in the performance of the product.

SHELF-LIFE

Shelf-life of dline Temporary Filling Material is 2 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

dline Temporary Filling Material is safe and performs as intended if it is used in accordance to manufacturer's instruction for use.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

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