



Knitted Retraction Cord



INSTRUCTION FOR USE

EN

DESCRIPTION

Knitted retraction cord is made of 100% cotton, knitted into thousands of tiny loops forming long interlocking chains. Knitted retraction cord is used in 4 sizes – #1 (for the anterior teeth preparation), #0 (for anterior teeth; as upper cord for “two cord” technique), #00 (during preparation and cementation of veneers; restorative procedures dealing with thin, friable tissues) and #000 (for lower anteriors; when luting near gingival and subgingival veneers; class III, IV V restorations; first cord for “two cord” technique). Marking depends on the thickness of cord. A unique attribute of this cord is that it doesn't entangle in the diamond bur.

For single use only.

INSTRUCTION FOR USE

NOTE! Do not use if primary package is broken.

1. Prepare and dry the tooth.
2. Choose the cord size. Because cord compresses upon packing, use a cord size that appears too large.
3. Cut the required length of retraction cord.
4. Soak the cord in the haemostatic liquid (not applicable for impregnated cord).
5. After preparing and drying the tooth the retraction cord is placed closely around the cervix and eased carefully into the sulcus using a small retraction cord packer or spatula.
6. The thin cord packer quickly slips it into position. The knitted cord's unique design (interlocking loops) facilitates easy packing and locks it into place.
7. During the following preparation for the procedures, the mechanical effect of the cord and chemical effect of hemostatic material, results in a definite retraction of the gingiva.
8. Perform the necessary procedures.
9. If any bleeding is present, repeat steps until hemostasis is achieved.
10. When the procedures are completed, remove cord and rinse with water.

WARNINGS AND PRECAUTIONS!

NOTE! Do not use dline Knitted Retraction Cord if patient is allergic to product or to any of the ingredients.

STORAGE

Keep container tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our devices have been developed for professional use in dentistry. As the application of our devices is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our devices in accordance with the applied standards.

PACKAGING

- | | |
|------------|---|
| REF | 24040 knitted retraction cord #1 (244cm), medium thick, blue |
| REF | 24010 knitted retraction cord #0 (244cm), medium, lilac |
| REF | 24020 knitted retraction cord #00 (244cm), thin, yellow |
| REF | 24030 knitted retraction cord #000 (244cm), ultra thin, black |

Made in EU for Dline OU
www.dline.ee



UAB "Medicinos linija"
Karaliaučiaus str. 29
LT-78348 Šiauliai
Lithuania

Last revised: 2019-06