


# dline Light Curing Pit And Fissure Sealant

**Light curing pit and fissure sealant for sealing  
small pits and fissures**

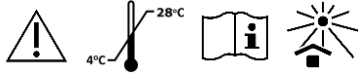

## INSTRUCTION FOR USE EN

### DESCRIPTION

dline Light Curing Pit And Fissure Sealant is white opaque, light curing, highly flowable sealant.

### INSTRUCTION FOR USE

**NOTE!** Do not use if primary package is broken. It is recommended to use cofferdam or other suitable protective measure during application of the product.

1. Prepare surface as always. Cleaning with oil-free prophylaxis paste, such as dline Prophy Paste or pumice is recommended.
2. Apply layer of etch, such as dline Phosphoric Acid Etching Gel on all surface to be etched. Leave etch in place for 30 seconds. Rinse with water and dry with air.
3. The working area must be kept dry and not contaminated.
4. Apply a thin layer of dline Light Curing Pit And Fissure Sealant immediately onto etched surface.
5. Twist a disposable dispensing tip onto the syringe. Lightly pressing the plunger of the syringe, apply sealant onto mixing pad. Then apply onto surface using disposable micro applicator.
6. Light curing for 20 seconds.
7. Inspect sealant for complete coverage and voids.
8. Wipe surface with cotton roll and check for hardness and retention.

**NOTE!** Sealant is highly flowable do not force bottle or syringe plunger. After the desired amount of sealant extruded, immediately close the bottle or the syringe.

### WARNINGS AND PRECAUTIONS!

**NOTE!** The pit and fissure sealant contains methacrylates. Uncured methacrylates may cause allergic reactions. Do not use dline Light Curing Pit And Fissure Sealant if patient is allergic to product or to any of the ingredients.

#### Hazard and precautionary statements:

Product causes skin, serious eye irritation, may cause respiratory irritation, and may cause an allergic skin reaction. **IF ON SKIN:** Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. **IF INHALED:** Remove victim to fresh air and keep at rest in a position comfortable for breathing. Call a POISON CENTRE or doctor/physician if you feel unwell.

#### Prevention:

Wash hands thoroughly after handling. Use only in a well-ventilated area. Wear protective gloves/protective clothing/eye protection/face protection.

#### Interaction with other substances:

dline Light Curing Pit And Fissure Sealant should not be used with products containing eugenol.

### STORAGE

Keep container tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze the product. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

### MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

### PACKAGING

<b>REF 15010</b>	3.8g syringe, 3 tips
<b>REF 15001</b>	4x1.2ml syringes, 10 tips
<b>REF 15000</b>	4x3.8g syringes, 12 tips

Made in EU for Dline OU  
www.dline.ee



UAB „Medicinos linija“  
Karaliaucius str. 29  
LT-78348 Siauliai  
Lithuania