

# dline Phosphoric Acid Etching Liquid



## INSTRUCTION FOR USE EN

### INDICATIONS

- Tooth enamel and / or dentin etching.

### DESCRIPTION

dline Phosphoric Acid Etching Liquid is 37% thixotropic phosphoric acid liquid with optimum level of viscosity.

### INSTRUCTION FOR USE

**NOTE!** Do not use if primary package is broken.

It is recommended to use cofferdam during application of the product.

1. Prepare surface as always. The working area must be kept dry.
2. If the surface to be etched is not freshly exposed, cleaning with oil-free prophylaxis paste such as dline Prophy Paste is recommended.
3. Apply layer of etch on all surface to be etched.
4. Leave etch liquid in place for 15 seconds (dentine), 30 seconds (enamel).
5. Rinse with water and dry with air.
6. Verify etched surface is clean and dry.
7. Use adhesive such as dline Light Curing Adhesive.

### WARNINGS AND PRECAUTIONS!

**NOTE!** Do not use dline Phosphoric Acid Etching Liquid if patient is allergic to product or to any of the ingredients.

#### Hazard and precautionary statements:

dline Phosphoric Acid Etching Liquid contains phosphoric acid, which causes skin corrosion, severe skin burns and eye damage. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. Wash contaminated clothing before reuse. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. Subsequently, have the casualty drink 1 - 2 glasses of water immediately. Do not try to neutralize with bases, no charcoal. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician.

#### Prevention:

Wash hands thoroughly after handling. Wear protective gloves / protective clothing / eye protection / face protection.

### STORAGE

Keep tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

### MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

### PACKAGING

**REF 20900** 10ml bottle

Made in EU for Dline OU  
www.dline.ee



UAB „Medicinos linija“  
Karaliaucius str. 29  
LT-78348 Siauliai  
Lithuania

Last revised: 2020-02