


# Dline Universal Handpiece Oil Spray



Dental handpieces heads oil

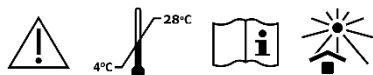

## INSTRUCTIONS FOR USE

EN

Hold bottle in a vertical position. Spray for one second into the bottom of handpiece after each use and before sterilization. Clean handpiece with paper towel.

### WARNINGS AND PRECAUTIONS!

Danger: Extremely flammable gas. Gases under pressure. Keep away from sources of ignition. Do not smoke. Do not breathe vapour-spray. Use only in well ventilated areas.

### STORAGE

Keep container tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight, sources of ignition and heat sources. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

### MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

### PACKAGING

**REF 25000** 500ml

Made in EU for Dline OU  
[www.dline.ee](http://www.dline.ee)



UAB „Medicinos linija“  
Karaliaučiaus str. 29  
LT-78348 Šiauliai  
Lithuania

Last revised: 2015-05