


Tooth Whitening Gel

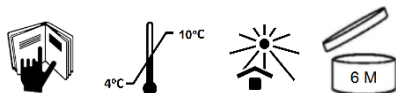

Ingredients: Aqua; Glycerin; Carbamide (Urea) peroxide; Aroma; Polyacrylic acid; Sodium hydroxide; Sodium fluoride; Sodium benzoate

INSTRUCTION FOR USE **EN** **DESCRIPTION**

dline Tooth Whitening Gel is a bleaching gel with fluoride for whitening vital, non-vital discoloured teeth. It is suitable for minimally invasive treatment using a custom-made bleaching tray. The dentist regulates the treatment and determines how long it should last.

DIRECTION FOR USE AT 23°C

NOTE! Do not use if primary package is broken. Not to be used on a person under 18 years of age. To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.

1. Remove the syringe cap. Insert a small amount of gel in the hollow tooth area of the bleaching tray. Do not fill with too much bleaching gel, as this does not improve or speed up the whitening process. One syringe is adequate for 8 to 10 applications. Once opened syringes can be stored closed with syringe cap.
2. Insert the tray with bleaching gel into the mouth and place over the teeth. Do not press the tray.
3. Remove excess material from the gums or palate using your finger or a toothbrush.
4. Wear the bleaching trays for at least 2 hours a day or overnight as instructed by your dentist.
5. Do not eat, drink or smoke while wearing the bleaching tray.
6. Remove the tray after procedure. Clean your teeth and rinse with water thoroughly to remove any residuals of gel. Clean tray thoroughly with warm water and a toothbrush. Dry it properly and store in the storage box.
7. Repeat the procedure as instructed by your dentist.

NOTE! Avoid highly acidic food and drink (cola, citrus drinks and fruit), as these can cause hypersensitivity of the tooth and oral mucosa. Strongly coloured drinks (coffee, tea, red wine) as well as smoking should be avoided during the whole procedures period, as these can retard the whitening process. Definite whitening can normally be seen after approximately 5 days. In the case of stubborn staining, your dentist could extend the period whitening procedures considerably.

Depending on the tooth structure and habits (consuming food containing colouring, smoking), whitening lasts for several of years. The procedure can be repeated later without any damage to the tooth structure. Consequently, the bleaching trays should be kept on completion of the procedures.

Tooth fillings as well as crowns, bridges and veneers made from porcelain, composite or metal are not bleached. They should be replaced if necessary and matched to the new, whiter tooth shade. Between bleaching and renewing any restorations there should be the period of 2 weeks.

In certain cases, these can be hypersensitivity of the teeth to hot and cold and mucosal irritation during and after procedures. These symptoms disappear if the amount of bleaching gel is reduced, or procedure is discontinued. Consult your dentist if the problems persist; if there are pressure points on the gums or the bleaching tray does not fit properly.

White spots may appear on the teeth during procedures. These are caused by transient demineralisation and disappear again.

WARNINGS AND PRECAUTIONS!

NOTE! Product contains carbamide peroxide and sodium fluoride. Contraindications: allergy to or intolerance of any of the ingredients; pregnancy; breast-feeding; smoking; dental caries.

Hazard and precautionary statements:

Avoid contact with the eyes, skin and mucosa. In contact, wash with plenty of water, if irritation persists: Get medical advice/attention. If an excessive amount of gel is swallowed, it can cause heat burn or a feeling of nausea in sensitive persons. Rinse mouth with water. Do not induce vomiting. Give one or two glasses of water to drink (ONLY IN CONSCIOUS PERSONS!). Refer for medical attention.

Prevention:

Wash hands thoroughly after handling. Wear protective gloves / protective clothing / eye protection / face protection.

STORAGE

Store product well closed in dark and dry place at 4-10°C. Protect from direct sunlight and heat sources. Do not freeze. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

PACKAGING

REF 26000	4x3g syringes, 10% Carbamide peroxide*, with fluoride***
REF 26010	4x3g syringes, 16% Carbamide peroxide**, with fluoride***

* Concentration of hydrogen peroxide: 3.3%

** Concentration of hydrogen peroxide: 5.3%

*** Contains sodium fluoride. Fluoride contents 0.10 wt%

MANUFACTURER:



Medicinos Linija UAB
Aviacijos str. 28
Siauliai LT-77103
Lithuania
Tel.: +370 41 553 553
www.dline.ee

Last revised: 2022-05